


Instructions Manual

Beauty instrument

F808.N

SKIN SCRUBBER

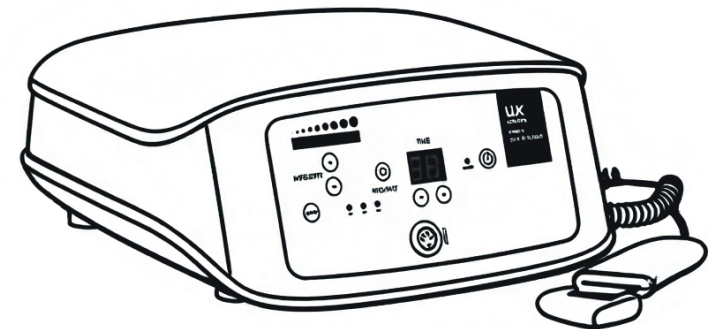

Imported by
WEEKKO BARCELONA SL

Pol. Ind. La Valldan
Cami de Garreta,
92

08600 Berga
Barcelona - Spain
T. +34 93 824 90

20
E.
info@weekko.com

VERSION 1.0

Thank you sincerely for choosing our product. This manual is compiled to help users operate this instrument effectively. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It serves as the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instrument's installation and various operation methods.

Usage Precautions

1. Before turning on the machine, be sure to read the user manual carefully. Operators must be familiar with the user manual and operation methods, and the instrument must be operated by a designated person;
2. When replacing the fuse, always turn off the power switch to prevent fire and
3. and
4. grounded
5. shock
6. in
7. vibrating,
8. case of
9. When installing the instrument, please contact the company for help;
10. When installing the instrument, it should be placed on a ventilated surface, with at least 5cm of
11. space left around it to ensure air circulation and facilitate heat dissipation of the instrument;
12. Do not cover the instrument with anything to prevent blockage of the air vents;
13. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnetic field environments; places where electromagnetic waves or impulse voltages occur; places where chemicals are stored or corrosive gases exist;

Product Packaging List

Name	Image	Quantity
Main Unit		1
Skin Scrubber Handle		1
Adapter		1
Power Cord		1

Product Specifications

Input Voltage: AC 220V-240V
Product Frequency: 50/60HZ
Product Power: 15W
Packaging Dimensions: 34x34x22CM
Gross Weight: 3.2KG
Net Weight: 2.7KG

- 14. This instrument shall not be placed or used near radios, televisions, copiers,
- 15. fax machines, etc.;
- Do not use it together with other instruments (such as microwaves) to
- 16. avoid
- mutual interference between instruments and cause misoperation;
- If an abnormal situation occurs to the instrument or the user during
- 17. operation,
- the operator shall immediately move the operation handle away from the
- 18. user, press the stop switch, and stop using the instrument;
- The instrument shall be correctly used by skilled operators. Personnel other
- 19. than operators are not allowed to use the instrument;
- After the operation, place the control handle back on the control handle holder;
- After using the operation handle, clean the probe thoroughly for next use.

Precautions for Troubleshooting, Storage, and Inspection

- 1. Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Please
- Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Please
- and
- and if the instrument is damaged, it should be inspected by a professional technician.
- 2. Do not use cleaning tools that may damage the instrument regularly.
- 3. protective
- 4. If the instrument is not used for more than 2 weeks, it must be
- 5. carefully
- inspected before reuse.

Contraindicated Groups


(Do Not Use If You Belong to the Following Groups)

Individuals with implanted cardiac pacemakers, defibrillators, or metal implants (Such as steel plates, bone nails) in the body;
Pregnant women during pregnancy;
Patients with epilepsy, hypertension, severe heart disease, sedation;
tumors or
trauma, skin diseases, skin damage, severe inflammation, eczema, dermatitis on the skin, or those with facial telangiectasia (spider veins)
Those who have received botulinum toxin injections, filler injections, laser treatments, chemical peels, or plastic surgery recently (usually within 6 months).

Important Safety Notes and Taboos

Frequency Control: Only for deep cleaning, use 1-2 times a week do not use
Daily Force: The technique must be gentle. Rely on the instrument's vibration
Keep Moist: During cleaning, the contact surface between the skin
Do not use on the face. Always use the same skin
Do not use on the face. Always use the same skin
Position for Repair: After use, immediately hydrate and soothe the skin (such as applying a facial mask or repair essence) and pay attention to sun protection.

Operation Instructions

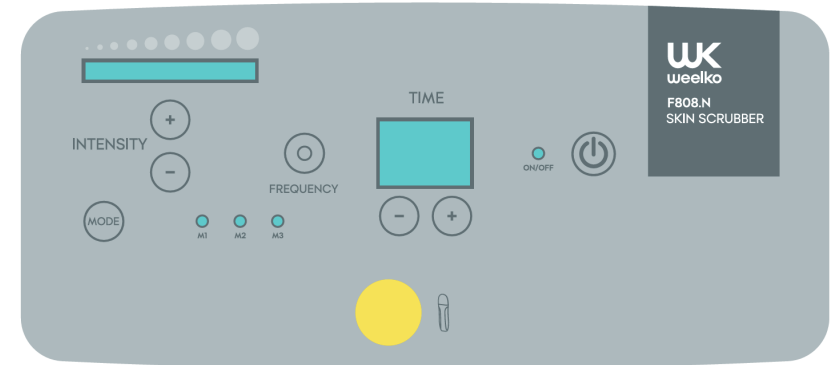

Instrument Operation Steps

1. Connect the device to a 220V power supply. The device will emit a "beep" sound, indicating that it is powered on and enters standby mode. The digital tube display will show 15 minutes, and the default energy is Level 1.
2. Press the '+' and '-' buttons on both sides of the digital tube display to control the working time. The setting range is 1-30 minutes, with a step size of 1 minute per adjustment.
3. Press the '+' and '-' buttons on both sides of the energy display bar to adjust the energy up or down. There are 1-10 gears adjustable, and Level 10 is the highest.
4. Press the 'MODE' button to search and lock the frequency;
5. Press the power button to start the skin scrubber function and begin the countdown. When the time is up, the device will emit a "beep-beep-beep~" prompt sound, the function will turn off, the time will reset to 15 minutes, and the energy will reset to Level 1.