


Instructions Manual

Beauty instrument

F812.N

ULTRASOUND & HIGH FREQUENCY

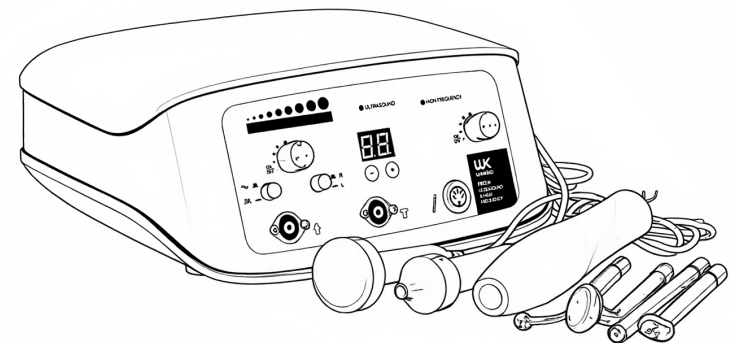

Imported by
WEEKKO BARCELONA SL

Pol. Ind. La Valldan
Camí de Garreta,
92

08600 Berga
Barcelona - Spain
T. +34 93 824 90

20
E.
info@weekko.com

VERSION 1.0

Thank you sincerely for choosing our product. This manual is compiled to help users operate this instrument ef_fectively. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It serves as the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instru-ment's installation and various operation methods.

Usage Precautions

1. Before turning on the machine, be sure to read the user manual carefully. Operators must be familiar with the user manual and operation methods, and the instrument must be operated by a designated person;
2. When replacing the fuse, always turn off the power switch to prevent f_ire
3. and
4. Electric shock. Power supply of 220V±22V, 50HZ±1HZ and an ef_fectively grounded
5. socket. A damaged or modif_ied power cord may cause f_ire and electric shock
6. If the instrument is loose, falling, or damaged, please contact the company in
7. time to use the instrument in unstable environments such as tilting, vibrating,
8. or in a wet environment. No water or other liquids are allowed to enter the interior of the instrument. In case of
9. When moving the instrument, please contact the company for help; f_irst;
10. When installing the host, it should be placed on a level surface, with at least 5cm of
11. space left around it to ensure air circulation and facilitate heat dissipation of the instrument;
12. Do not cover the instrument with anything to prevent blockage of the air vents;
13. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnet-ic f_ield environments; places where electromagnetic waves or impulse voltag-es occur; places where chemicals are stored or corrosive gases exist;


Product Packaging List

Name	Image	Quantity
Main Unit		1
High-Frequency Handle		1
Tongue-Shaped Electrotherapy Tube		1
Straight Electrotherapy Tube		1
Curved Electrotherapy Tube		1
Mushroom-Shaped Electrotherapy Tube		1
Facial Ultrasonic Probe		1
Eye Ultrasonic Probe		1
Power Cord		1

Product Specif_ications

Input Voltage: AC 220V-240V
Product Frequency: 50/60HZ
Product Power: 40W
Packaging Dimensions: 34x34x22CM
Gross Weight: 3.4KG
Net Weight: 2.9KG

Instrument Introduction

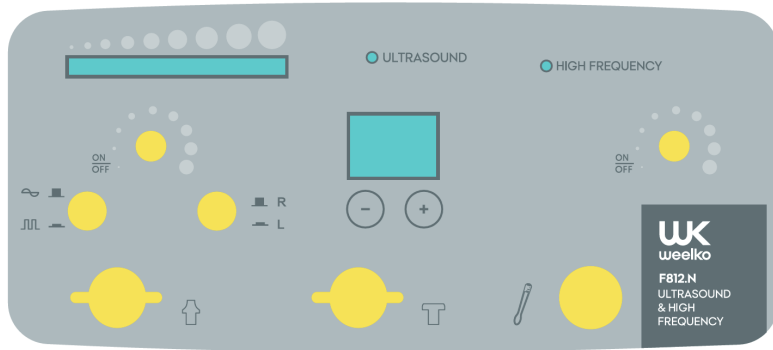

Instrument Operation Steps

1. Connect the device to a 220V power supply. The device will emit a "beep" sound, indicating that it is powered on and enters standby mode. The digital tube displays the set time of 30 minutes, the default function is High Frequency, the default energy is 0 gear, and the energy bar is fully of; the selected electrotherapy tube before use;
2. Turn the  (HIGH FREQUENCY) switch. The  (HIGH FREQUENCY) light will turn on, activating the high-frequency function. Adjust the energy from low to high using the knob, insert the ultrasonic probe into the corresponding two positions. The  light will turn on. Use the adjustment switch to control the power and energy level;
3. Press the  (P) button to select 2 ultrasonic modes: continuous output and intermittent; Press the  (T) button to switch between the facial probe and the eye probe;
4. After setting, start the corresponding function for treatment. Follow the operation techniques and duration requirements in the respective function sections;
5. After the operation, turn off the power, remove the probes and electrotherapy tubes, clean them thoroughly, and store the instrument in a dry and cool place.

14. This instrument shall not be placed or used near radios, televisions, copiers, fax machines, etc.;
15. Do not use it together with other instruments (such as microwaves) to avoid mutual interference between instruments and cause misoperation;
16. If an abnormal situation occurs to the instrument or the user during operation, the operator shall immediately move the operation handle away from the user, press the stop switch, and stop using the instrument;
17. The instrument shall be correctly used by skilled operators. Personnel other than operators are not allowed to use the instrument;
18. After the operation, place the control handle back on the control handle holder;
19. After using the operation handle, clean the probe thoroughly for next use.

Precautions for Troubleshooting, Storage, and Inspection

1. Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Immediately press the stop button, turn off the power, unplug the power plug, and contact the professional distributor immediately; prohibited from repairing or modifying the instrument. The interior of the instrument must be inspected regularly; Do not use cleaning tools that may damage the instrument;
2. protective
3. If the instrument is not used for more than 2 weeks, it must be carefully inspected before reuse.

Contraindicated Groups

(Do Not Use If You Belong to the Following Groups)

Individuals with implanted cardiac pacemakers, defibrillators, or metal implants (such as steel plates, bone nails) in the body; pregnant women during sedation; Patients with epilepsy, hypertension, severe heart disease, tumors or brain tumors, thyroid diseases, skin damage, severe inflammation, eczema, dermatitis on the skin, or those with facial telangiectasia (spider veins) or those who have received botulinum toxin injections, filler injections, laser treatments, chemical peels, or plastic surgery recently (usually within 6 months).

Core Components

The instrument mainly consists of an insulated handle and multiple replaceable

glass electrode tubes (called "electrotherapy tubes" in the packaging list).

Glass Electrotherapy Tubes and Their

Uses: Mushroom-Shaped Tube: Suitable for large-area areas such as cheeks and forehead;

Curved Tube/Tongue-Shaped Tube/Straight Tube: Suitable for sensitive or small-area parts such as eye area, lip area, and nasal alae.

Operation Process

1. Pre-Use Preparation

Inspect the Instrument: Ensure the instrument is dry and intact, and the glass

electrotherapy tube is clean and free of cracks; ensure the treatment area is

clean and free of cosmetics; First select and insert the appropriate glass electro-

therapy tube. Apply a sufficient amount of special conductive gel or essence on the skin. Dry operation is strictly prohibited to provide conductivity and avoid burns.

2. Operation Process

Power On and Adjustment: Turn on the power and be sure to start from the

lowest intensity gear. Usually, rotating clockwise increases the intensity, and rotating counterclockwise decreases it;

Avoid Dangerous Areas: Strictly prohibit use on eyeballs, throat, open wounds, severe acne or infected areas.

3. Post-Use Operation

Shutdown Sequence: First adjust the intensity knob back to the lowest gear or

turn off the power. After the power is off and unplugging the power, carefully

cleaning and disinfecting the electrode tube. Do not pull it apart; and electrotherapy

tube with a soft damp cloth, and store them in a cool and dry place after drying.