


Instructions Manual

Beauty instrument

F834.N

DIAMOND DERMABRASION

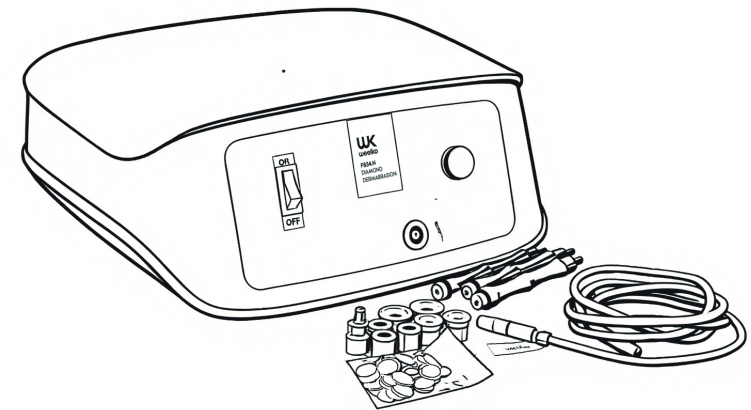

WK

Imported by
WEEKKO BARCELONA SL

Pol. Ind. La Valldan
Cami de Garreta,
92

08600 Berga
Barcelona - Spain
T. +34 93 824 90

20
E.
info@weekko.com

VERSION 1.0

Thank you sincerely for choosing our product.. This manual is compiled to help users operate this instrument ef_fectively. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It serves as the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instru-ment's installation and various operation methods.

Usage Precautions

1. Before turning on the machine, be sure to read the user manual carefully. Operators must be familiar with the user manual and operation methods, and the instrument must be operated by a designated person;
2. When replacing the fuse, always turn off the power switch to prevent f_ire and
3. and
4. grounded
5. shock
6. in
7. vibrating,
8. case of
9. f_irst;
10. When installing the host, be placed on a level surface, with at least 5cm of
11. space left around it to ensure air circulation and facilitate heat dissipation of the instrument;
12. Do not cover the instrument with anything to prevent blockage of the air vents;
13. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnet-ic f_ield environments; places where electromagnetic waves or impulse voltag-es occur; places where chemicals are stored or corrosive gases exist;

Product Packaging List

Name	Image Quantity
Main Unit	1
Dermabrasion Handle	1 box
Transparent Suction Tube	1
Power Cord	1

Product Specifications

Input Voltage: AC 220V-240V

Product Frequency: 50/60HZ

Product Power: 60W

Packaging Dimensions: 34x34x22CM

Gross Weight: 3.5KG

Net Weight: 3.0KG

14. This instrument shall not be placed or used near radios, televisions, copiers,
15. fax machines, etc.;
- Do not use it together with other instruments (such as microwaves) to
16. avoid mutual interference between instruments and cause misoperation;
- If an abnormal situation occurs to the instrument or the user during
17. operation,
- the operator shall immediately move the operation handle away from the
18. user, press the stop switch, and stop using the instrument;
- The instrument shall be correctly used by skilled operators. Personnel other
19. than operators are not allowed to use the instrument;
- After the operation, place the control handle back on the control handle holder;
- After using the operation handle, clean the probe thoroughly for next use.

Precautions for Troubleshooting, Storage, and Inspection

1. Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Immediately press the stop button, turn off the power, unplug the power plug, and contact the professional distributor immediately; prohibited
2. from repairing or modifying the instrument. The interior of the instrument must be inspected regularly; Do not use cleaning tools that may damage the instrument;
3. protective
4. if the instrument is not used for more than 2 weeks, it must be
5. carefully inspected before reuse.

Contraindicated Groups
(Do Not Use If You Belong to the Following Groups)

Individuals with implanted cardiac pacemakers, defibrillators, or metal implants (Such as steel plates, chains, pins) in the body; men during Pregnancy or lactation; Patients with epilepsy, hypertension, severe heart disease, arrhythmia; tumors or benign or malignant diseases; skin damage, severe inflammation, eczema, dermatitis on the skin, or those with facial telangiectasia (spider veins) Those who have received botulinum toxin injections, filler injections, laser treatments, chemical peels, or plastic surgery recently (usually within 6 months);

Important Safety and Maintenance Instructions

Force and Duration: During operation, be sure to be gentle and slow. Rely on the instrument's own functions; do not press hard or stay in the same position for a long time. Maintenance: The device is for personal use only. It is recommended to replace it regularly to avoid bacterial growth. Store the instrument in a dry and cool place.



Instrument Operation Steps

1. Connect the device to a 220V power supply and press the  power button. The device will emit a "beep" sound, indicating that it is powered on and in standby mode;
2. Insert the suction tube into the filter nozzle, connect the suction pen  to the suction tube, and insert the selected dermabrasion head. Press the small hole  on the handle to use it normally;
3. Adjust the suction power from low to high using the  knob.