


Instructions Manual

Beauty instrument

WKB015

SKIN SCRUBBER

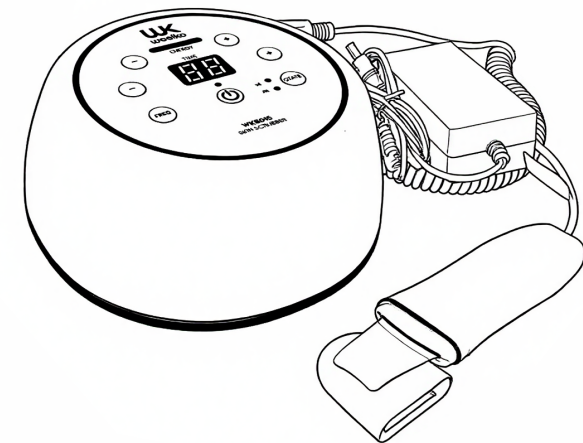

WK
Imported by
WEEKO BARCELONA SL

Pol. Ind. La Vall d'an
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weeko.com

VERSION 1.0

Thank you for choosing our product! This manual is designed to help you make the most of this instrument, and we hope our products and services will bring you a distinguished experience.

This user manual contains all the general information about this instrument. It serves as a starting point for new users, providing a general overview of the system, explaining how it works, and describing the installation process and various operating methods.

Precautions for Use

1. Always read the instruction manual carefully before turning on the machine. The operator must be familiar with the manual and operating procedures, and the instrument must be managed by designated personnel.
2. Please do not use power strips to avoid fire hazards.
3. Always turn off the power switch when replacing a fuse to prevent fire or electric shock.
4. The instrument must be used with a 220V $\pm$ 22V, 50Hz $\pm$ 1Hz power supply and a properly grounded socket.
5. Using damaged or modified power cords poses risks of fire and electric shock.
6. If the instrument becomes loose, detached, or damaged, please contact our company immediately.
7. Do not use this instrument in unstable environments such as tilting, vibrating, or impact prone areas.
8. Water or foreign objects must not enter the instrument. If any foreign object enters, please contact our company immediately.
9. The power plug must be unplugged before moving the instrument; avoid vibration as much as possible during movement.
10. The instrument should be placed on a stable surface with at least 5cm of space around it to ensure air circulation and facilitate heat dissipation.
11. Do not cover the instrument with anything to prevent blockage of the ventilation openings.
12. Avoid using this instrument in the following environments: direct sunlight; high humidity; near water sources; vibrating or tilting locations; strong magnetic field environments; areas with electromagnetic waves or impulse voltages; places storing chemicals; and areas with corrosive gases.
13. This instrument must not be placed or used near radios, televisions, copiers, fax machines, etc.

Product Packaging List


Name	Image Quantity
Main Unit	1
Skin Scrubber	1
Adapter	1
Power Cord	1

Product Specifications

- Input voltage: AC220V-240V
- Product frequency: 50/60Hz
- Product power: 15W
- Packaging dimensions: 27×22×15cm
- Gross weight: 1.4kg
- Net weight: 1.15kg

Instrument Introduction

Product control panel

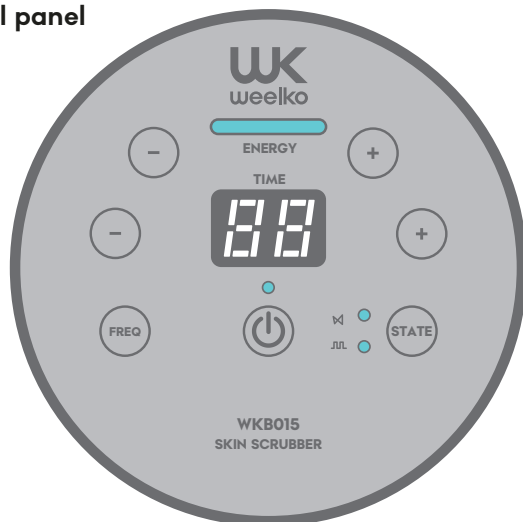

Operation Instructions

1. When the device is connected to a 220V power supply, it will emit a "beep" sound, indicating that the device is powered on and has entered standby mode. The digital display shows a preset time of 15 minutes, and the default energy level is Level 1.
2. Press the left (+) or right (-) buttons on the digital time display window to adjust the working time. The setting range is 1-30 minutes, with each adjustment step being 1 minute.
3. Press the left (+) or right (-) buttons on the energy display bar to adjust the high-frequency energy or import/export energy level. (Total of 1-10 intensity levels adjustable, with Level 10 being the highest intensity.)
4. Press the (STATE) button to switch the peeler's working mode: (continuous mode) or (intermittent mode).
5. Press the (FREQ) button to search for and lock the frequency.
6. Press the (power) button to activate the peeler function and begin the count-down. When the time is up, the device will emit a "beep-beep-beep" sound, the function will turn off automatically, the time will reset to 15 minutes, and the energy will reset to Level 1.

14. Do not use it in conjunction with other instruments (such as microwaves) to avoid mutual interference and malfunctions.
15. If any abnormal situation occurs with the instrument or the user during operation, the operator shall immediately move the operating handle away from the user, press the stop switch, and cease using the instrument.
16. This instrument must be correctly used by skilled operators. Personnel other than skilled operators are not allowed to use this instrument.
17. After operation, place the control handle back into the control handle bracket.
18. After using the operating handle, clean the probe in preparation for the next use.

Precautions for Troubleshooting, Storage, and Inspection

1. Do not use the instrument if it malfunctions. If it emits smoke, produces a burning smell, or becomes inoperable, continued use may cause fire or electric shock. Please immediately press the stop button, turn off the power, unplug the power cord, and contact our company or distributor right away.
2. No one other than professional technicians may repair or modify this instrument. Under no circumstances should the interior of the instrument be opened.
3. The instrument and its components must be inspected regularly.
4. Do not use cleaning tools that may damage the instrument's protective coating, such as brushes or brooms.
5. If the instrument has not been used for more than 2 weeks, it must be carefully inspected before being used again.

Contraindications

(The following individuals should not use this product)

1. Individuals with pacemakers, defibrillators, or metal implants (such as plates or bone screws) in their bodies.
2. Pregnant women, breastfeeding women, and women during menstruation.
3. Individuals with epilepsy, hypertension, severe heart disease, diabetes, tumors or cancer, or thyroid disease.
4. Those with open wounds, lesions, severe inflammation, eczema, dermatitis, facial redness (telangiectasia), or rosacea.
5. Individuals who have recently (usually within 6 months) undergone Botox injections, filler injections, laser treatments, chemical peels, or plastic surgery.

Working Principle and Operation Guide

This skin-scraping beauty instrument uses high-frequency ultrasonic vibrations (typically 28,000 times per second) to loosen and emulsify dirt and oil on the skin's surface and in pores, which are then gently removed through physical scraping. It is primarily used for deep pore cleansing, removing oil, dirt, and dead skin cells; some products also have an absorption-enhancing function to help skincare products penetrate better.

(I) Deep Cleansing Mode (Blackhead Removal/Exfoliation)

1. Shovel head direction: Convex side (or cutting edge) facing upward, in contact with the skin.
2. Preparation: Thoroughly cleanse your face. Before use, open pores by applying a hot towel or using a facial steamer. For optimal results, use in conjunction with a blackhead-removing solution as a compress (e.g., on the nose). Ensure the skin is moist (apply water or the blackhead-removing solution); do not use on dry skin.
3. Operating technique: Hold the spatula head at a 45-degree angle to the skin and gently glide it slowly against the direction of the pores (usually from bottom to top). You will hear a "buzzing" vibration sound when it is working properly.
4. Usage time: It is recommended to cleanse the entire face for 5-7 minutes. For sensitive skin, the time can be shortened to 3 minutes.

(II) Nutrient Import Mode (if this function is available)

1. Shovel head direction: Convex side facing downward, flat against the skin.
2. Preparation: After cleansing, apply a sufficient amount of serum, gel, or facial mask to your face.
3. Technique: Hold the spatula at a small angle (approximately 20 degrees) to the skin and gently glide or circle it from the inside out, following the direction of the muscle texture.
4. Usage time: Recommended to be around 5 minutes.

Important Safety Precautions and Prohibitions

1. Frequency control: For deep cleansing purposes only, use 1-2 times per week; do not use daily.
2. Pressure control: The operating technique must be gentle, relying on the instrument's vibration to function. Do not press or scrape forcefully.
3. Keep moist: During cleansing, ensure that both the skin and the contact surface of the spatula head remain moist to prevent friction-induced skin damage.
4. Continuous movement: Do not leave the spatula head stationary on the same spot of the skin for an extended period.
5. Post-treatment care: Immediately after use, moisturize and soothe the skin (e.g., apply a facial mask or repair serum) and ensure adequate sun protection.