


Instructions Manual

Beauty instrument

F803.N

GALVANIC & HIGH FREQUENCY

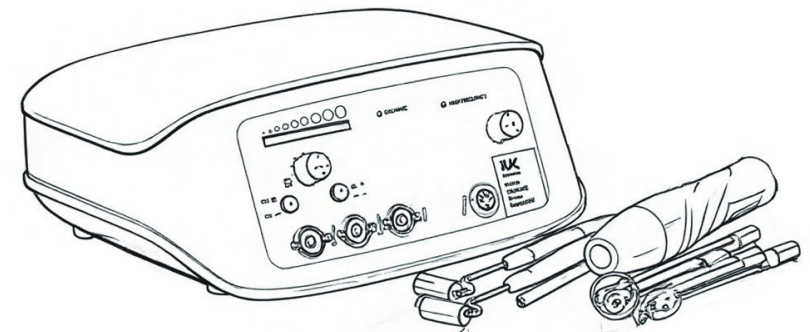

Imported by
WEEKKO BARCELONA SL

Pol. Ind. La Valldan
Camí de Garreta,
92

08600 Berga
Barcelona - Spain
T. +34 93 824 90

20
E.
info@weekko.com

VERSION 1.0

Thank you sincerely for choosing our product. This manual is compiled to help users operate this instrument effectively. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It serves as the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instrument's use, precautions and various operation methods.

1. Before turning on the machine, be sure to read the user manual carefully. Operators must be familiar with the user manual and operation methods, and the instrument must be operated by a designated person;
2. When replacing the fuse, always turn off the power switch to prevent fire and
3. and
4. Electric shock. Power supply of 220V±22V, 50HZ±1HZ and an effectively grounded
5. socket. A damaged or modified power cord may cause fire and electric shock
6. If the instrument is loose, falling, or damaged, please contact the company in
7. time to use the instrument in unstable environments such as tilting, vibrating,
8. or water dripping. Do not allow water to enter the interior of the instrument. In case of
9. When moving the instrument, please contact the company properly;
10. When installing the instrument, it should be placed on a flat surface, with at least 5cm
11. of space left around it to ensure air circulation and facilitate heat dissipation of the instrument;
12. Do not cover the instrument with anything to prevent blockage of the air vents;
13. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnetic field environments; places where electromagnetic waves or impulse voltages occur; places where chemicals are stored or corrosive gases exist;

Product Packaging List

Name	Image Quantity
Main Unit	1
Galvanic Roller Handles	2
Galvanic Handheld Stick	1
High-Frequency Handle	1
Tongue-Shaped Electrotherapy Tube	1
Straight Electrotherapy Tube	1
Curved Electrotherapy Tube	1
Mushroom-Shaped Electrotherapy Tube	1
Power Cord	1

Product Packaging List

Input Voltage: AC 220V-240V

Product Frequency: 50/60HZ

Product Power: 15W

Packaging Dimensions: 34x34x22CM

Gross Weight: 3.2KG

Net Weight: 2.8KG

14. This instrument shall not be placed or used near radios, televisions, copiers,
 15. fax machines, etc.;
 - Do not use it together with other instruments (such as microwaves) to
 16. avoid mutual interference between instruments and cause misoperation;
 - If an abnormal situation occurs to the instrument or the user during
 17. operation,
 - the operator shall immediately move the operation handle away from the
 18. user, press the stop switch, and stop using the instrument;
 - The instrument shall be correctly used by skilled operators. Personnel other
 19. than operators are not allowed to use the instrument;
 - After the operation, place the control handle back on the control handle holder;
 - After using the operation handle, clean the probe thoroughly for next use.
- Precautions for Troubleshooting, Storage, and Inspection**

1. Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Immediately stop the operation, unplug the power cord, and contact the company or distributor immediately;
2. repairing or modifying the instrument. The interior of the instrument must be inspected by a professional technician. The instrument and its components must be inspected before use. Do not use cleaning tools that may damage the instrument's surface.
3. Do not use the instrument if the instrument is not used for more than 2 weeks, it must be carefully inspected before reuse.

Contraindicated Groups

(Do Not Use If You Belong to the Following Groups)

1. Individuals with implanted cardiac pacemakers, defibrillators, or metal implants (such as steel plates, chains, nails) in the body; men during pregnancy and women during lactation;
2. Patients with epilepsy, hypertension, severe heart disease, or diabetes;
3. tumors or
4. bruising, skin damage, severe inflammation, eczema, dermatitis on the skin, or those with facial telangiectasia (spider veins);
5. Those who have received botulinum toxin injections, filler injections, laser treatments, chemical peels, or plastic surgery recently (usually within 6 weeks);

Operation Process

1. Pre-Use Preparation

Inspect the Instrument: Ensure the instrument is dry and intact, and the glass electrode is not cracked. Clean the skin thoroughly and ensure the treatment area is dry.

Connect the Electrodes: First select and insert the appropriate glass electrode.

Apply Medium: Apply a sufficient amount of special conductive gel or essence on the skin. Dry operation is strictly prohibited to provide conductivity and avoid burns.

2. Post-Use Operation


Shutdown Sequence: First adjust the intensity knob back to the lowest gear or

Remove the Electrodes: After the power is off and unplugging the power, carefully

Clean and Store: Clean the glass electrode and the treatment handle, and

electrotherapy tube with a soft damp cloth, and store them in a cool and dry place after drying.

Instrument Operation Steps

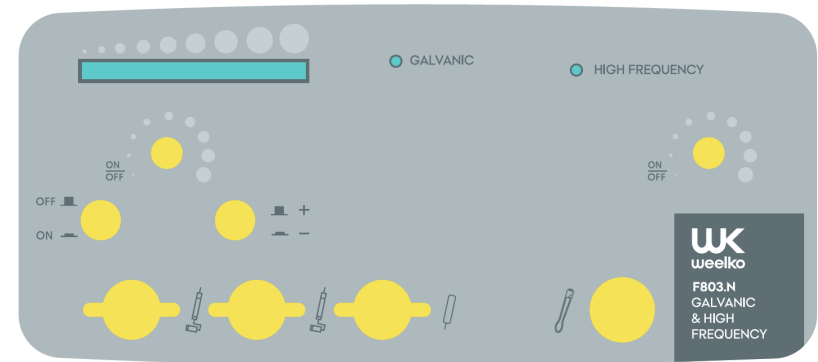

Instrument Operation Steps

1. Connect the device to a 220V power supply. The device will emit a "beep" sound, indicating that it is powered on and enters standby mode;
2. Insert the Galvanic accessory handles according to the marks  Insert the high-frequency handle according to the accessory mark, and insert the selected electrotherapy tube before use;
3. Use the  switch to turn on/off the Galvanic function;
4. Press the  button to switch between Galvanic Import/Export modes;
5. Adjust the Galvanic energy from low to high using the knob. There are 10 energy levels with corresponding indicator lights;
6. Use the  (HIGH FREQUENCY) switch to turn on/off the high-frequency function, and adjust the energy from low to high;
7. After setting, start the corresponding function for treatment. Follow the operation techniques and duration requirements in the respective function sections;
8. After the operation, turn off the power, remove the handles and electrotherapy tubes, clean them thoroughly, and store the instrument in a dry and cool place.