


Instructions Manual

Beauty instrument

F829.N

SPOT REMOVAL

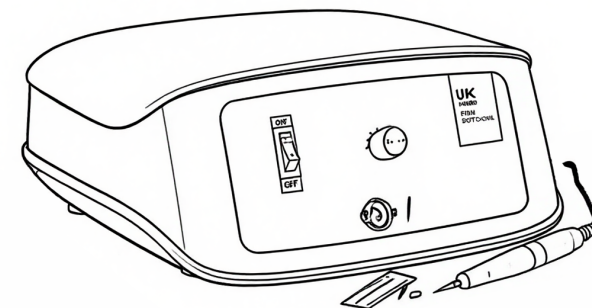

Imported by
WEEKKO BARCELONA SL

Pol. Ind. La Valldan
Cami de Garreta,
92

08600 Berga
Barcelona - Spain
T. +34 93 824 90

20
E.
info@weekko.com

VERSION 1.0

Thank you sincerely for choosing our product. This manual is compiled to help users operate this instrument effectively. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It serves as the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instrument's installation and various operation methods.

Usage Precautions

1. Before turning on the machine, be sure to read the user manual carefully. Operators must be familiar with the user manual and operation methods, and the instrument must be operated by a designated person;
2. When replacing the fuse, always turn off the power switch to prevent fire and hazards;
3. and
4. Electric shock power supply of 220V±22V, 50HZ±1HZ and an effectively grounded socket; a damaged or modified power cord may cause fire and electric shock;
5. If the instrument is loose, falling, or damaged, please contact the company;
6. in
7. Do not use the instrument in unstable environments such as tilting, vibrating, or tilting;
8. No water or other liquids are allowed to enter the interior of the instrument. In case of
9. When moving the instrument, please contact the company properly;
10. When installing the host, it should be placed on a ventilated surface, with at least 5cm of
11. space left around it to ensure air circulation and facilitate heat dissipation of the instrument;
12. Do not cover the instrument with anything to prevent blockage of the air vents;
13. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnetic field environments; places where electromagnetic waves or impulse voltages occur; places where chemicals are stored or corrosive gases exist;

Product Packaging List

Name	Image Quantity
Main Unit	1
Spot Removal Pen	1
Needle Holder	1
Acupuncture Needle	1
Mole Removal Needle	1
Power Cord	1

Product Specifications

Input Voltage: AC 220V-240V

Product Frequency: 50/60HZ

Product Power: 10W

Packaging Dimensions: 34x34x22CM

Gross Weight: 2.85KG

Net Weight: 2.35KG

14. This instrument shall not be placed or used near radios, televisions, copiers,
15. fax machines, etc.;
- Do not use it together with other instruments (such as microwaves) to
16. avoid mutual interference between instruments and cause misoperation;
- If an abnormal situation occurs to the instrument or the user during
17. operation,
- the operator shall immediately move the operation handle away from the
18. user, press the stop switch, and stop using the instrument;
- The instrument shall be correctly used by skilled operators. Personnel other
19. than operators are not allowed to use the instrument;
- After the operation, place the control handle back on the control handle holder;
- After using the operation handle, clean the probe thoroughly for next use.


Precautions for Troubleshooting, Storage, and Inspection

1. Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Immediately press the stop button, turn off the power, unplug the power plug, and contact the professional distributor immediately; prohibited from repairing or modifying the instrument. The interior of the instrument must be inspected regularly; Do not use cleaning tools that may damage the instrument;
2. protective
3. if the instrument is not used for more than 2 weeks, it must be carefully inspected before reuse.

Contraindicated Groups
(Do Not Use If You Belong to the Following Groups)

Individuals with implanted cardiac pacemakers, defibrillators, or metal implants (Such as steel plates, chains, nails) in the body, men during Pregnancy or lactation, Patients with epilepsy, hypertension, severe heart disease, sedation; tumors or bruise, skin damage, severe inflammation, eczema, dermatitis on the skin, or those with facial telangiectasia (spider veins) Those who have received botulinum toxin injections, filler injections, laser treatments, chemical peels, or plastic surgery recently (usually within 6 months).

Instrument Introduction

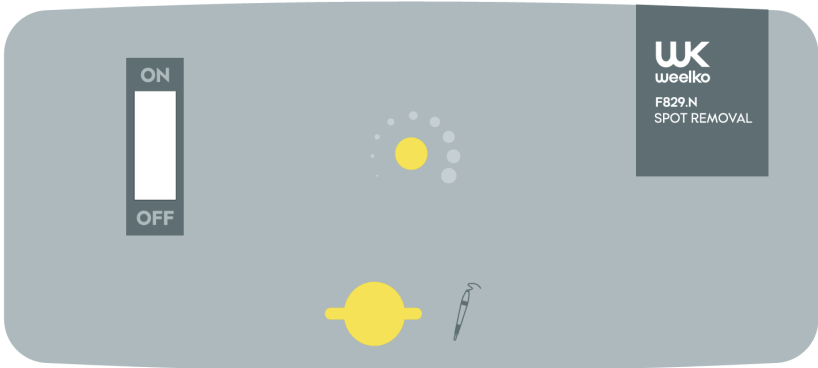

Instrument Operation Steps

1. Connect the device to a 220V power supply and turn on the ON/OFF switch . The device will emit a "beep" sound, indicating that it is powered on and enters standby mode;
2. Insert the spot removal pen into the output hole . There are two types of pens: thick needle and thin needle. Select and use according to needs;
3. Adjust the energy output from low to high using the knob;
4. After setting the energy, start the operation following the requirements in the "Operation Process" section;
5. After the operation, turn off the power, remove the spot removal pen, clean and disinfect the needle, and store the instrument in a dry and cool place.

