MultiEquipment. Ref F-330

Manual de instrucciones

5 EN 1

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Instruction manual

5 IN 1

Magic Glove  
Bio Skin Smoother  
Skin scrubber  
Ultrasound  
Galvanic

---

## Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

---

## Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

---

## EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

---

## Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

**weelko**  
the wellness  
company

Importer:


Weelko Barcelona S.L  
Pol. Ind. La Valldan  
Camí de Garreta, 92  
08600 Berga  
Barcelona – Spain  
T. +34 938 24 90 20  
E. info@weelko.com  
W. www.weelko.com  
ESB65397556

## Front panel

## Instructions of use

1. Connect the power supply via the rear panel.
2. Connect the accessories properly to the front/rear panel as indicated in the sketch.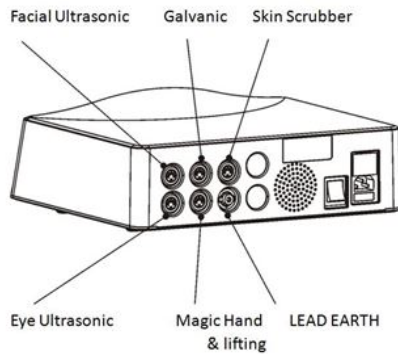

3. Turn on the "POWER" button.
4. Press "FUNCTION" to select the desired function in the following order: Skin Scrubber, Ultrasonic, Galvanic, Magic Hand & Lifting.

## Notes

A positive earth control system is in place while the machine is operating. This means that unless the machine can find an earth return, the system will stop the current output to the probe. However, once the probe comes back into contact with the customer (the earth circuit is found), the system will restart the output immediately. This is a safety feature and should not be mistaken for a system fault.

## SKIN SCRUBBER

1. Place the bracelet around the customer's wrist.
2. Apply the appropriate conductive product generously to the surface of the skin.
3. Select the desired wave type by pressing "STATE", the desired output energy by pressing "ENERGY" and time of working by pressing "TIMER". You can change setting anytime during working.
4. Press "START/PAUSE" button to start. Once there is circuit between the client and the scrubber, machine will start to work. The Scrubber sign on the screen will run when machine is working normally. Once there is separation between the scrubber and the client, machine will pause, and the Scrubber sign on the screen will stop running.

## Cautions

- The ones with the following diseases are required not to use it: heart disease, infectious disease, a person with metal in body due to treatment, a person with extremely sensitive physical structure.
- Do not keep the probe at the same position of the skin too long to avoid burning.
- In case the exclusive liquid is insufficient, supplement it.
- As the probe is fitted and unloaded, please do it in accordance with due order. Do not pull it off with force.
- As the power wire is disconnected from the back, pull it off from the socket. Do not pull its wire off.
- Do not hit the horn of the probe or damage the stainless steel surface to avoid skin scratch.

---

## **ULTRASOUND**

### **Before treatment**

1. Cleanse/tone the treatment area using the prescribed skin-type products.
2. Remove any left-over product by facial steam for 10 minutes.
3. Cleanse the treatment area again to remove any dirt or debris from the face in order to prevent the ultrasound waves from pushing it back into the dermis.
4. Tone every part of the face (or treatment area) gently using a suitable astringent.
5. After wiping off the astringent, the skin will be thoroughly cleansed for optimum protection.
6. Apply enough conductive gel to the treatment area.

### **Starting treatment**

1. Select the desired ultrasonic probe by pressing "SELECT", wave type by pressing "STATE", the desired output energy by pressing "ENERGY" and time of working by pressing "TIMER". You can change setting anytime during working.
2. Press "START/PAUSE" button to start.

---

## **Notes**

- Always make sure enough conductive gel is applied for at least 15 minutes of treatment time.
- Generally, lower outputs are used for facials and higher outputs for body treatments.
- If the treatment area is small or sensitive, use the eye probe and make sure that the energy output is limited to 5-7 blocks (on the "ENERGY" LCD screen display). The treatment time may also be reduced to 8-10 minutes.
- A standard treatment course lasts around 10 sessions in total, with a minimum resting period of 1 day between sessions. If other facial treatments are being conducted on the same area, a minimum rest period of 7 days should be observed between treatments.

---

## **Cautions**

- Never use on dry skin.
- Do not use on the stomach area of pregnant women.
- Do not use on people with pace-makers (any electrical or rhythmic heart-regulating device).
- Do not use on people with known heart conditions.
- Always use ultrasonic electrodes with enough moisturizer or serum on the skin.
- Ultrasonic treatment is suitable for all skin types. The massage time and machine settings can be adjusted for each different skin type. For sensitive skins, reduce treatment time to 3-5 minutes and make sure the power setting is at the lowest level to start with.

---

## **GALVANIC**

### **Before treatment**

1. The treatment area must be washed and dried before any products can penetrate into the tissue.
2. The polarity of the probes can be changed by adjusting the "STATE" button between (+R) or (-G).
3. Cover the roller/ball with a piece of cotton or a sponge that has been dipped into a suitable lotion (according to the treatment). The lotion can also be applied directly to the skin.

### **Starting treatment**

1. Choose the desired galvanic probes (roller or ball). Then properly connect two identical probes in the wires.
2. Select the desired polarity by pressing "STATE", the power output level must be at the minimum and select the desired time of working by pressing "TIME". You can change any of these settings during the treatment.
3. Press the "START/PAUSE" button.

---

## **Contraindications**

- Pacemaker (any electrical or rhythmic heart-regulating device)
- Pregnancy
- Thrombosis/Phlebitis
- Skin diseases
- Operations (within the last 6 months)
- Silicone implants
- Hypersensitive or vascular skin

---

## **Cautions**


- Always make sure customers remove any jewelry, and confirm that they have not had any metal implants (pins, joints, etc.) installed since their last treatment.
- Pregnant women and people with sensitive, injured, inflamed skin or blood capillary expansions should not receive this treatment.
- Customers with health issues such as heart disease or infectious diseases and hyper-allergic or hyper-sensitive skin (see contra-indications) may not use this instrument.
- Always make sure the skin surface has been moistened with plenty of conductive product.

---

## MAGIC GLOVES

### Instructions of use

1. Connect one side of the wire in the rear panel, in the plug named "BIO WAVE MICROCURRENT".
2. Connect the other two sides of the wires in each glove.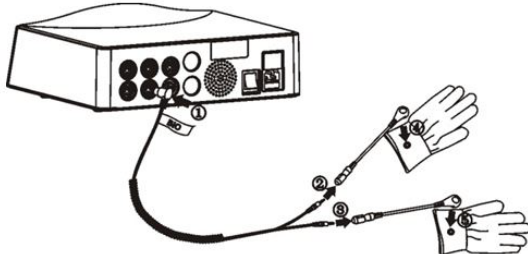

3. The beautician must put isolating gloves under the magic gloves.
3. Select the magic glove function by pressing "FUNCTION" button.
3. Make sure that the "ENERGY" level is at the minimum. You can adjust the output level by pressing "ENERGY +/-".
3. Make sure that the "FREQUENCY" level is adjusted at 0.1 Hz. You can adjust the frequency level by pressing "SATE".
3. Select the desired wave state by pressing "SELECT". There are 4 kinds of waves.
3. Select the time of operating by pressing "TIMER +/-".
3. Press "START/PAUSE" to start the treatment.
3. The beautician must massage to beautician with the magic gloves. Adjust the "ENERGY" and "FREQUENCY" level according to the customer feelings. At this moment, the beautician don't have to feel any effect in their hands, but the customer must feel a micro current and some warm in the part where the treatment are applied.

---

## Notes

- The delivered magic gloves have a length of 20 cm. The isolating gloves must have a length of at least 23 cm. If there are contact between the beautician's skin and the magic gloves, there's no effect with the treatment.
- During the treatment, if the beautician breaks the contact with the customer's skin, the device stops working suddenly. The device restarts when the contact between the magic gloves and the customers skin returns.

---

## Cautions


- Pacemaker (any electrical or rhythmic heart-regulating device)
- Pregnancy
- Thrombosis/Phlebitis
- Skin diseases
- Operations (within the last 6 months)
- Silicone implants
- Hypersensitive or vascular skin

---

## BIO-LIFTING

### Instructions of use

1. Connect one side of the wire in the rear panel, in the plug named "BIO WAVE MICROCURRENT".
2. Connect the other two sides of the wires in the handles of BIO-lifting. There are two kinds of handles.
  - The ones that need cotton swabs are used in the eyes part. It is needed to include two cotton swabs in the hole of the handle.
  - The ones with only one pole are used to the full body.

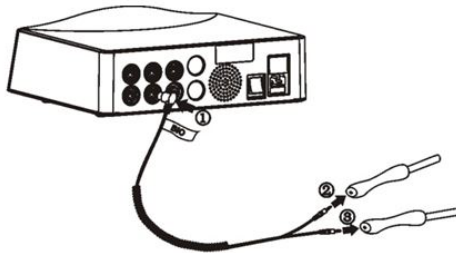3. Apply enough conductive gel to the treatment area.
4. Make sure that the "ENERGY" level is at the minimum. You can adjust the output level by pressing "ENERGY +/-" button.
5. Make sure that the "FREQUENCY" level is adjusted at 0.1 Hz. You can adjust the frequency level by pressing "RATE".
6. Select the desired wave state by pressing "SELECT". There are 4 kinds of waves.
7. Select the time of operating by pressing "TIMER +/-".
8. Press "START/PAUSE" to start the treatment.

---

## FAQ

### Skin scrubber

The probe suddenly stops emitting:

- If the probe is separated from the skin, it will stop emitting after 5 seconds. This is a safety feature and should not be mistaken for a system fault.
- Touch the skin with the probe and it will start emitting again after 5 seconds.

### Ultrasound

Check the following if the work unit switches on but the probes do not emit.

- Put a drop of water on one probe.
- Switch on the unit and slowly increase the output energy.
- If the water starts to move and evaporate, the unit and the probes are working properly. If not, the probes are not working properly.

The unit and the probes are working properly but the customer doesn't feel any effects.

- Make sure enough conductive gel has been applied to the treatment area.

---

## **MAINTENANCE**

### **Skin scrubber**

- Any oil on the probe must be removed immediately.
- After use, please clean the probe and keep it probe. Store it in the probe-protecting sleeve at all time.
- Store the probe carefully to avoid damaging it.

### **Ultrasound**

- The ultrasound probes should be cleaned using clean water (not hot) and a clean, soft cloth and then chemically sterilized using an appropriate sterilization fluid. Dry the probe carefully after sterilizing and store it safely in its holder on the trolley in between uses. Always sterilize the probes again prior to use.
- Never immerse the probe in any fluid!

### **Galvanic**

- Take care not to damage the probe rollers. Never use abrasive substances to clean the probe rollers, since damage to the surface of the roller may scratch the surface of the skin during treatment; any damaged areas may also harbor bacteria and make sterilization more difficult.
- After treatment, all metal roller parts must be cleaned, sterilized and dried. Then store them in the accessory holder to avoid any damage.

### **Magic gloves & BIO-lifting**

- After each treatment, the magic gloves and the BIO-lifting handles must be cleaned, sterilized and properly saved in a dry place.

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