MultiEquipment. Ref F-332

Manual de instrucciones

4 EN 1

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Diamond microdermabrasion
Ultrasound
Cold & Hot Hammer
Skin Scrubber

Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

weelko
the wellness
company

Importador/Importer
Importateur/Importeur
Importatore:

Weelko Barcelona, S.L.
ESB65397556

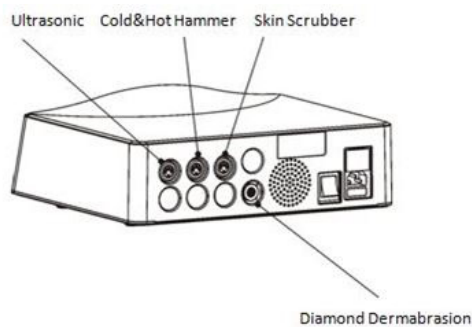
Barcelona
Pol. Ind. La Vall dan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 938 24 90 20
E. info@weelko.com
W. www.weelko.com

Front panel

Instructions of use

1. Connect the power supply via the rear panel.
2. Connect the accessories properly to the front/rear panel as indicated in the sketch.

3. Turn on the "POWER" button.
4. Press "FUNCTION" to select the desired function in the following order: Skin Scrubber, Ultrasonic, Diamond Dermabrasion, Cold & Hot Hammer.

Notes

A positive earth control system is in place while the machine is operating. This means that unless the machine can find an earth return, the system will stop the current output to the probe. However, once the probe comes back into contact with the customer (the earth circuit is found), the system will restart the output immediately. This is a safety feature and should not be mistaken for a system fault.

SKIN SCRUBBER

1. Place the bracelet around the customer's wrist.
2. Apply the appropriate conductive product generously to the surface of the skin.
3. Select the desired wave type by pressing "STATE", the desired output energy by pressing "ENERGY" and time of working by pressing "TIMER". You can change setting anytime during working.
4. Press "START/PAUSE" button to start. Once there is circuit between the client and the scrubber, machine will start to work. The Scrubber sign on the screen will run when machine is working normally. Once there is separation between the scrubber and the client, machine will pause, and the Scrubber sign on the screen will stop running.

Cautions

- The ones with the following diseases are required not to use it: heart disease, infectious disease, a person with metal in body due to treatment, a person with extremely sensitive physical structure.
- Do not keep the probe at the same position of the skin too long to avoid burning.
- In case the exclusive liquid is insufficient, supplement it.
- As the probe is fitted and unloaded, please do it in accordance with due order. Do not pull it off with force.
- As the power wire is disconnected from the back, pull it off from the socket. Do not pull its wire off.
- Do not hit the horn of the probe or damage the stainless steel surface to avoid skin scratch.

ULTRASOUND

Before treatment

1. Cleanse/tone the treatment area using the prescribed skin-type products.
2. Remove any left-over product by facial steam for 10 minutes.
3. Cleanse the treatment area again to remove any dirt or debris from the face in order to prevent the ultrasound waves from pushing it back into the dermis.
4. Tone every part of the face (or treatment area) gently using a suitable astringent.
5. After wiping off the astringent, the skin will be thoroughly cleansed for optimum protection.
6. Apply enough conductive gel to the treatment area.

Starting treatment

1. Select the desired ultrasonic probe by pressing "SELECT", wave type by pressing "STATE", the desired output energy by pressing "ENERGY" and time of working by pressing "TIMER". You can change setting anytime during working.
2. Press "START/PAUSE" button to start.

Notes

- Always make sure enough conductive gel is applied for at least 15 minutes of treatment time.
- Generally, lower outputs are used for facials and higher outputs for body treatments.
- If the treatment area is small or sensitive, use the eye probe and make sure that the energy output is limited to 5-7 blocks (on the "ENERGY" LCD screen display). The treatment time may also be reduced to 8-10 minutes.
- A standard treatment course lasts around 10 sessions in total, with a minimum resting period of 1 day between sessions. If other facial treatments are being conducted on the same area, a minimum rest period of 7 days should be observed between treatments.

Cautions


- Never use on dry skin.
- Do not use on the stomach area of pregnant women.
- Do not use on people with pace-makers (any electrical or rhythmic heart-regulating device).
- Do not use on people with known heart conditions.
- Always use ultrasonic electrodes with enough moisturizer or serum on the skin.
- Ultrasonic treatment is suitable for all skin types. The massage time and machine settings can be adjusted for each different skin type. For sensitive skins, reduce treatment time to 3-5 minutes and make sure the power setting is at the lowest level to start with.

DIAMOND DERMABRASION

1. Correctly connect one side of the pipe to the rear panel as indicated in the sketch.
2. Connect the opposite side of the pipe to the vacuum handle. Place the correct diamond head in the handle. There are nine types of diamond heads.

Ø 22 MM	
D100	Body. Suitable for acne, knees, elbows and areas with stretch marks
D180	
Ø 19 MM	
D180	Face. Suitable for large spots, pores, aging sun exposure
D220	
Ø 15 MM	
D100	Scars, fine lines and various imperfections
D200	
D250	
Ø 8 Y 6 MM	
D200	Eyes and both sides of the nose. Wrinkles, acne, pimples and blackheads
D250	

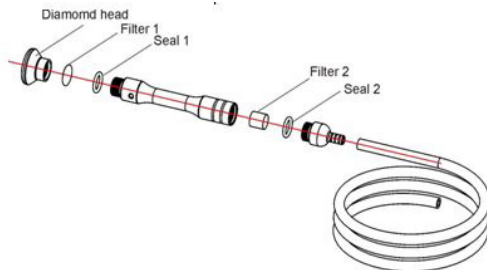3. Check that the foam filter is in its proper position in the handle.
4. Check that the metal filter is in its proper position in the handle.

5. You can adjust the suction effect by covering or uncovering the hole in the side of the handle with your finger.
6. Select the desired energy and time. You can change setting anytime during working.
7. Press "START/PAUSE" button to start.

Cautions

- Do not leave the handle on the same part of skin for too long since it can cause injuries.
- Avoid using it on irritated skin.
- The following skin areas are unsuitable for these treatments: skin tumors, moles, birthmarks, angiomas, lips, eyes, herpes and warts.

HOT & COLD HAMMER

1. Press "STATE" to select between cold or hot function.
2. According to the needs, adjust the output temperature by pressing "ENERGY" and desired time of operation by pressing "TIMER".
3. Press "START/PAUSE" to start working.

Notes

- For first time usage, function will start 5 minutes after all set up.
- After the first treatment, it needs 3 minutes before the next Cold or Hot treatment.
- Best working temperature condition is 25C.
- Do not block the air tank while using this function, if not steaming water will be blocked inside the Cold & Hot hammer, and may induce short circuit.
- Even after the "START/PAUSE" button has been turned on to "START", but need to check if the hammer has been switched to "o", if still in "-", hammer would not start working.

Cautions

- The 2 sides of Cold & Hot Hammer are made of Aluminum Alloy which is not hard, avoid being knocked on them.
- If there is cracks on the 2 sides of hammer, replace with new one
- Be careful of the elastic wire working extension, do not over pull it.

FAQ

Skin scrubber

The probe suddenly stops emitting:

- If the probe is separated from the skin, it will stop emitting after 5 seconds. This is a safety feature and should not be mistaken for a system fault.
- Touch the skin with the probe and it will start emitting again after 5 seconds.

Ultrasound

Check the following if the work unit switches on but the probes do not emit.

- Put a drop of water on one probe.
- Switch on the unit and slowly increase the output energy.
- If the water starts to move and evaporate, the unit and the probes are working properly. If not, the probes are not working properly.

The unit and the probes are working properly but the customer doesn't feel any effects.

- Make sure enough conductive gel has been applied to the treatment area.

Diamond dermabrasion

If the suction effect is reduced:

- Replace the metal and foam filters.

MAINTENANCE

Skin scrubber

- Any oil on the probe must be removed immediately.
- After use, please clean the probe and keep it probe. Store it in the probe-protecting sleeve at all time.
- Store the probe carefully to avoid damaging it.

Ultrasound

- The ultrasound probes should be cleaned using clean water (not hot) and a clean, soft cloth and then chemically sterilized using an appropriate sterilization fluid. Dry the probe carefully after sterilizing and store it safely in its holder on the trolley in between uses. Always sterilize the probes again prior to use.
- Never immerse the probe in any fluid!

Diamond dermabrasion

- The foam filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- The metal filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- Clean and sterilize the diamond heads after every treatment.

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