ADVERTENCIA

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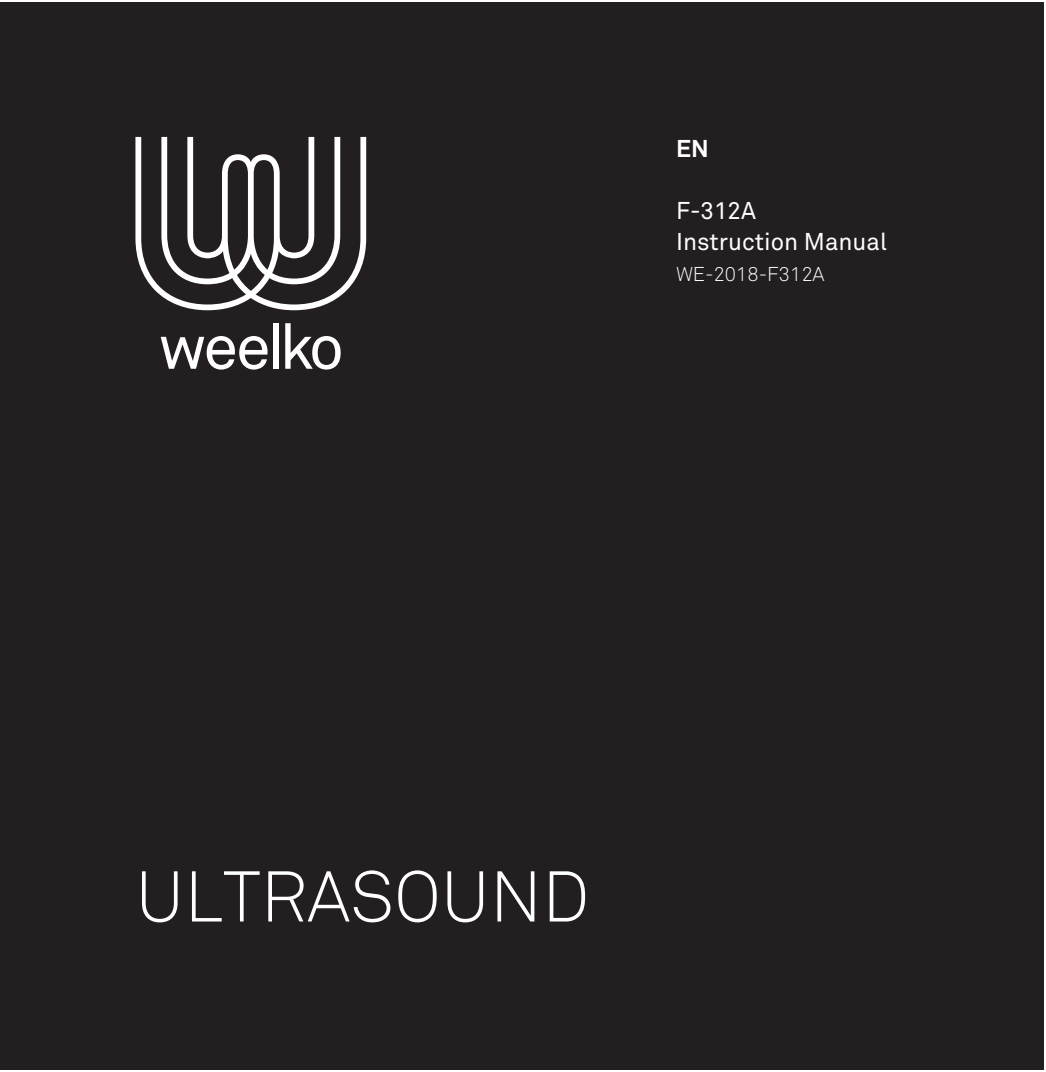
WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

• Do not leave any ultrasound probes switched on whilst unattended, otherwise they may overheat, which could result in damage to the probe or a reduction of its service life.

• Do not attempt to separate the stainless-steel generator from its rubber holder (handle); any attempt to do so may cause irreparable damage to the generator.

EN
F-312A
Instruction Manual
WE-2018-F312A

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

- 1. Conditions set out in the Low Voltage Directive: 2014/35/EU
- 2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!




Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to

incorrect operation. The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.


PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.(CONTINGUT CAIXA ENG)

No.	PRODUCT	REF.	QTY.
1	EYE PROBE	10034	1
2	FACE PROBE	10035	1
3	MAIN UNIT	-	1
4	FEEDER CABLE	-	1

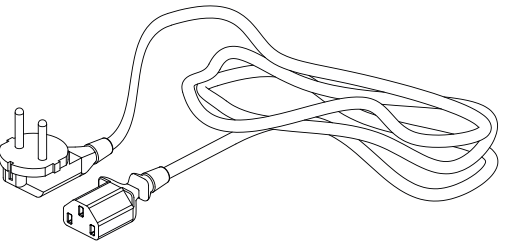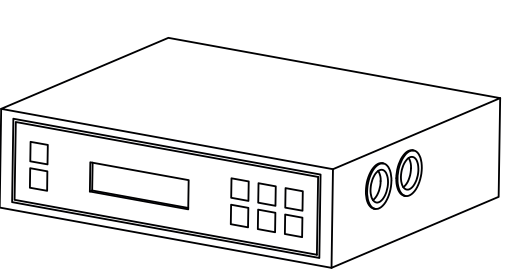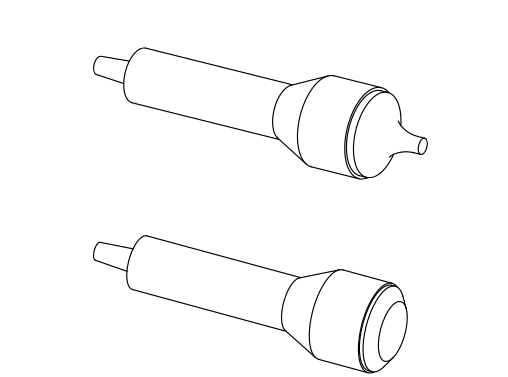FRONT PANEL

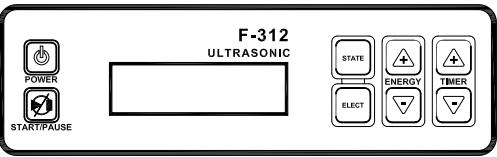INSTRUCTIONS OF USE

BEFORE TREATMENT:

1. Cleanse/tone the treatment area using the prescribed skin-type products.
2. Remove any left-over product by facial steam for 10 minutes.
3. Cleanse the treatment area again to remove any dirt or debris from the face in order to prevent the ultrasound waves from pushing it back into the dermis.
4. Tone every part of the face (or treatment area) gently using a suitable astringent.

5. After wiping off the astringent, the skin will be thoroughly cleansed for optimum protection.

6. Apply enough conductive gel to the treatment area.

STARTING TREATMENT:

1. Connect the power supply via the rear panel, indicated in the sketch as “POWER LINE INPUT”.
2. Connect the accessories properly in the front/rear/lateral panel as indicated in the sketch.
3. Turn on the “POWER” button.
4. Adjust the intensity of the output according to the “ENERGY” blocks on the LCD display. Generally, you should start at 3-4 blocks and then increase as required.

5. Select the desired wave state by switching the “STATE” button. There are 3 kinds of wave state: continuous wave, intermittent wave, and combined wave.

6. Select the desired output probe (face or eyes) by pressing “SELECT”.

7. Hold the ultrasonic probe firmly and press the “START/PAUSE” button to begin.

8. Massage the treatment area where the conductive gel has been applied using small circular motions to cover the whole treatment area. Treatment time will vary depending on the size of treatment area. A standard (complete) facial area should take 10-15 minutes of repetitive massage.

9. After completing the treatment, switch off the output and return the probe to its holder.

NOTES

- Always make sure enough conductive gel is applied for at least 15 minutes of treatment time.
- Generally, lower outputs are used for facials and higher outputs for body treatments.

• If the treatment area is small or sensitive, use the eye probe and make sure that the energy output is limited to 5-7 blocks (on the “ENERGY” LCD screen display). The treatment time may also be reduced to 8-10 minutes.

• A standard treatment course lasts around 10 sessions in total, with a minimum resting period of 1 day between sessions. If other facial treatments are being conducted on the same area, a minimum rest period of 7 days should be observed between treatments.

CAUTIONS

- Never use on dry skin.
- Do not use on the stomach area of pregnant women.
- Do not use on people with pace-makers (any electrical or rhythmic heart-regulating device).

• Do not use on people with known heart conditions.

- Always use ultrasonic electrodes with enough moisturizer or serum on the skin
- Ultrasonic treatment is suitable for all skin types. The massage time and machine settings can be adjusted for each different skin type. For sensitive skins, reduce treatment time to 3-5 minutes and make sure the power setting is at the lowest level to start with.

FAQ

Check the following if the work unit switches on but the probes do not emit.

- Put a drop of water on one probe.
- Switch on the unit and slowly increase the output energy.
- If the water starts to move and evaporate, the unit and the probes are working properly. If not, the probes are not working properly.
- Make sure enough conductive gel has been applied to the treatment area.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	37x27,5x14
	Gross weight (kg)	3,5
	Product size (cm)	20,7x22x7
Working characteristics	Working power	0 - 3 W/cm2
	Working frequency	1 MHz
	Preset working time	15 min
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	20 W


MAINTENANCE

• The ultrasound probes should be cleaned using clean water (not hot) and a clean, soft cloth and then chemically sterilized using an appropriate sterilization fluid. Dry the probe carefully after sterilizing and store it safely in its holder on the trolley in between uses. Always sterilize the probes again prior to use.

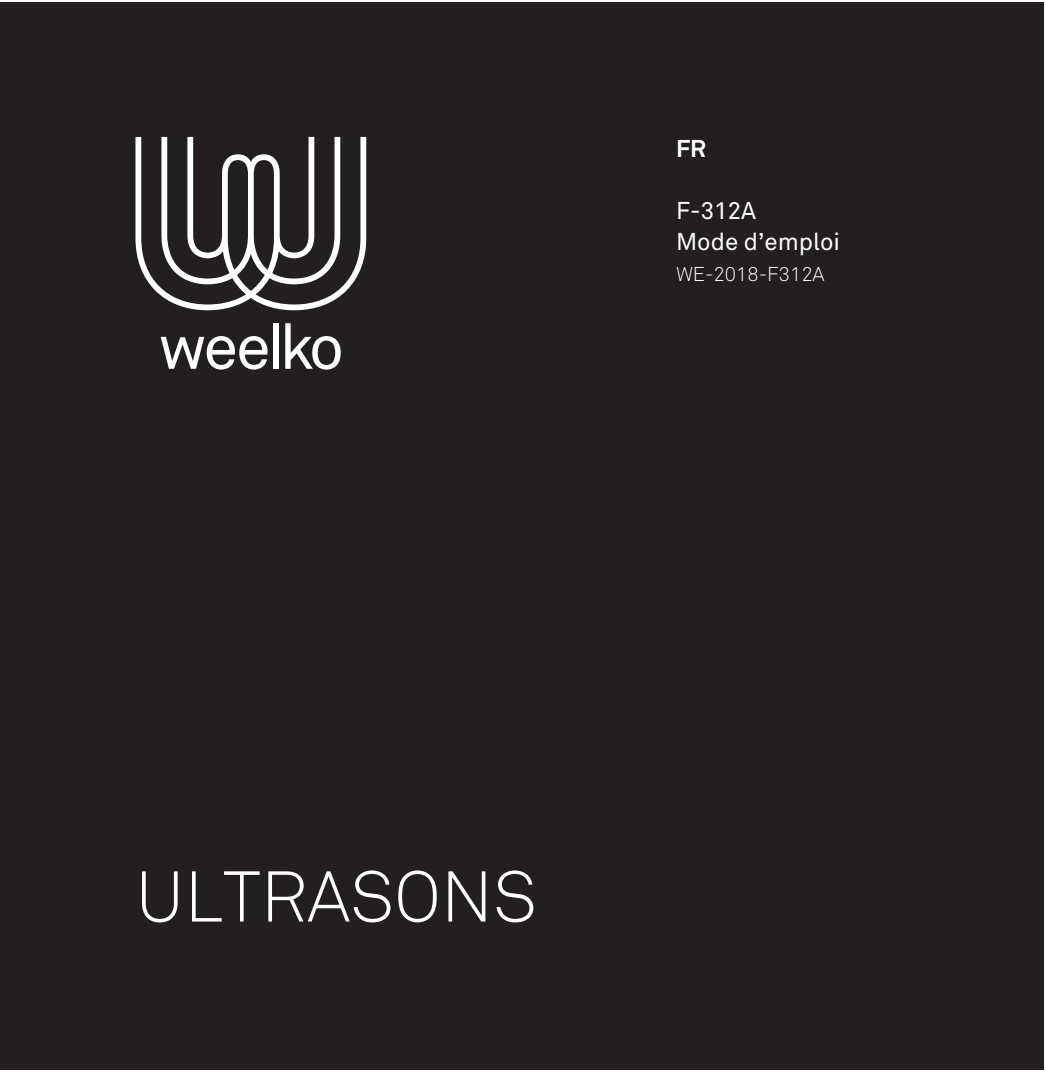• Never immerse the probe in any fluid!

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