B-Equipment. Ref F-318A

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

Martillo de frío y calor
Cold & hot hammer
Marteau froid et chaud
Kälte & wärme hammer
Martello freddo & caldo

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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Instructions of use

1. Connect the power supply via the rear panel.
2. Turn on the "POWER" button.
3. Press "STATE" to select between cold or hot function.
4. According to the needs, adjust the output temperature by pressing "ENERGY" and the desired time of operation by pressing "TIMER".
5. Press "START/PAUSE" to begin the treatment.

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Notes

1. For first time usage, the function will start 5 minutes after all the set up process.
2. After the first treatment, it needs 3 minutes before the next Cold or Hot treatment.
3. Best working temperature condition is 25C.
4. Do not block the air tank while using this function, if not, steaming water will be blocked inside the Cold & Hot hammer, and may induce short circuit.
5. Even after the "START/PAUSE" button has been turned on to "START", you may need to check if the hammer has been switched to "-", if it is still in "o", the hammer would not start working.

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Caution

1. The two sides of Cold & Hot Hammer are made of aluminum alloy which is not hard, so avoid knocking on them.
2. If there is cracks on the sides of hammer, replace it with a new one
3. Be careful with the elastic wire working extension, do not over pull it.

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Maintenance

1. Do not disinfect the plastic parts and the wire with chemical products such as gasoline, alcohol or other medical elements, because it may cause the fading of cover and may affect the function
2. Clean the body of hammer and the aluminum parts with a soft cloth after the operation and then store them properly
3. If there's any scratch on the sides of the hammer, please replace it with a new one before operating.

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