ADVERTENCIA

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SPOT REMOVAL

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!




Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.



PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

| No. | PRODUCT             | REF.  | QTY. |
|-----|---------------------|-------|------|
| 1   | SPOT REMOVAL HANDLE | 10039 | 1    |
| 2   | MAIN UNIT           | -     | 1    |
| 3   | FEEDER CABLE        | -     | 1    |

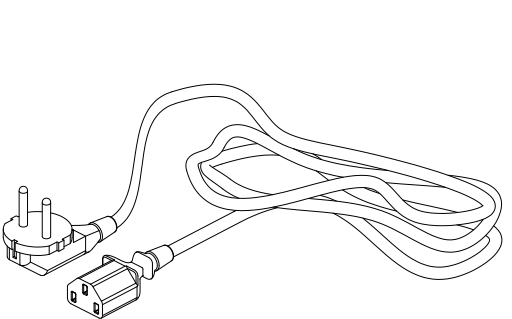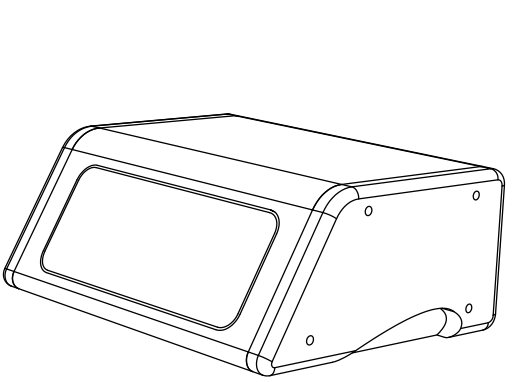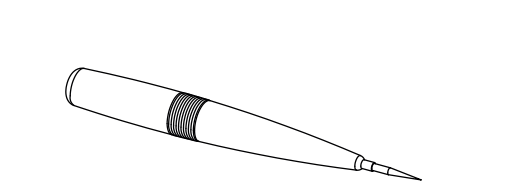
FRONT PANEL/REAR PANEL

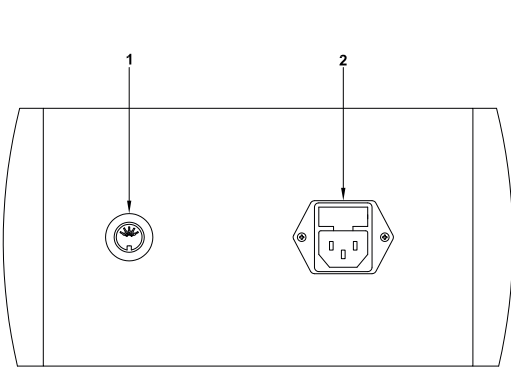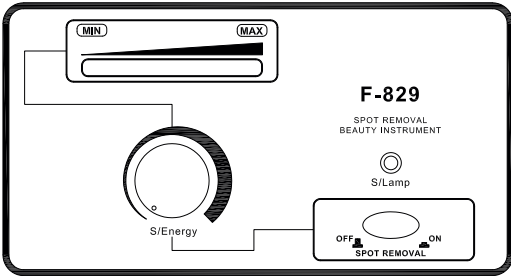1. Spot removal handle
2. Power line input

INSTRUCTIONS OF USE

1. Connect the power supply via the rear panel, indicated in the sketch as “POWER LINE INPUT”.
2. Connect the accessories properly to the front/rear panel as indicated in the sketch.

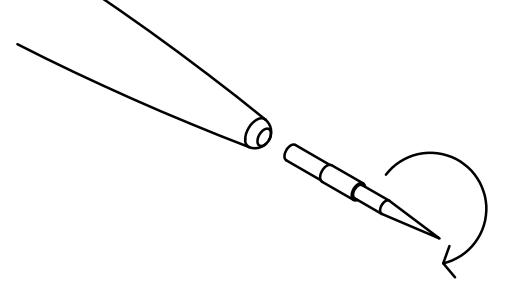3. Press the “SPOT REMOVAL” button.
4. Turn the “R/ENERGY” regulator to the desired level.
5. Hold the spot removal pen as if you were holding a pen. Aim it at the spot on the surface and move it in gradually until the pen point emits a spark towards the spot. Remove the pen quickly. If the spots are not completely removed, repeat the above steps. If the spot is too big, remove it in several goes.

CAUTIONS

- Due to the varying thickness of the cutin layer, the ideal results are different when removing spots. It is advisable to use the test and touch method to remove the spot.
- A small red mark will be left after removing spots, which is a normal reaction.
- A small scar will be left after removing spots. Do not remove it using your finger or nail in order to avoid infection.

ACCESSORIES

| REF.  | DESCRIPTION         |
|-------|---------------------|
| 10039 | SPOT REMOVAL HANDLE |

- Daub an adequate repairing essence to promote skin growth and renewal after removing a spot.
- Be sure to turn the energy level to the lowest setting after using the instrument, in order to avoid burning the body when using it the next time.

MAINTENANCE

- Please disinfect the head of pen before and after spot removal.

TECHNICAL SHEET

|                            |                   |             |
|----------------------------|-------------------|-------------|
| Weights and sizes          | Packing size (cm) | 36x31x18,5  |
|                            | Gross weight (kg) | 3,1         |
|                            | Product size (cm) | 31x17/24x13 |
| Electrical characteristics | Rated voltage     | 220V ~ 240V |
|                            | Frequency         | 50Hz/60Hz   |
|                            | Rated power       | 15 W        |

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