

# F-362 / F-363 / F-364 / F-365

## Beauty Instrument

### User Manual

Welcome to use our products. A0  
Please read the user manual carefully before using the products.  
Our company reserves the right to improve design and modify  
the products without prior notice.



About this manual	1
Warning	1
Product parameter and introduction of F-362	2/3
Product parameter and introduction of F-363	4/5
Product parameter and introduction of F-364	6/7
Product parameter and introduction of F-365	8/9
Hot and cold hammer introduction	10/11
Ultrasonic probe introduction	12/13
Skin Scrubber introduction	14/15
Radio Frequency introduction	16/17
Diamond Dermabrasion introduction	18/19
Vacuum/Spray introduction	20/21
High frequency electrotherapy introduction	22/23
Rotary Brush introduction	24/25
Galvanic introduction	26/27
RES & CAP Operation Precautions	28
RES Introduction	29
CAP Introduction	30
Emergency measures and Troubleshooting	31

## About This Manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, SILVERFOX will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of our company SILVERFOX.

## Safety Guidance

**To ensure your correct and safe use of this product, please read all the instructions before use!  
Please follow all safety precautions!**

	<b>Warning!</b>	<p><b>Indicates a potential dangerous situation that, if not avoided, could result in serious loss of life.</b></p> <ul style="list-style-type: none"><li>• This product is only for beauty practitioners with professional qualifications.</li><li>• Do not use or store the product in a damp place such as bathroom to avoid malfunction and danger.</li><li>• Do not use this product near flammable items (e.g., alcohol, thinner, gasoline, nail polish remover, sprays, etc.).</li><li>• Do not use this product under the following conditions: when the power cord or plug is damaged or gets hot; when the plug in the power socket comes loose.</li><li>• Do not use wet hands to plug and unplug power cord connectors and use this product.</li><li>• Please make sure that the working voltage of this product conforms to its rated voltage.</li><li>• Before use, make sure that the power plug and the power cord connector are fully plugged in, so as not to cause danger.</li><li>• When not using this product, please turn off the power and pull out the plug.</li><li>• Please regularly clean the dust accumulated on the power plug and power cord connector to avoid failure of insulation caused by moisture, dust and other factors.</li><li>• Be sure to use the special power cord assembly for this product. Do not use the power cord assembly on any other appliance.</li><li>• Do not wrap the power cord around the product, otherwise it may distort and malfunction the power cord.</li><li>• Keep out of reach of children. Do not allow children to use this product.</li><li>• Please do not modify, disassemble or repair this product by yourself. If you need to repair it, please contact the manufacturer, agent or professional maintenance personnel to check and repair it, so as not to cause danger.</li><li>• Please contact distributor, service center or related personal with appropriate skill and experience for repairing if the plug is damaged, in case of any danger.</li><li>• Please be sure to use this product according to the purpose specified in this manual. Do not use accessories not recommended by the manufacturer.</li></ul>
	<b>Attention!</b>	<p><b>Represents a potentially dangerous situation which, if not avoided, can result in general injury to person.</b></p> <ul style="list-style-type: none"><li>• Operators should pay special attention to the safety of personnel and equipment when using.</li><li>• If the product is abnormal, please stop using it immediately and contact the manufacturer or dealer for relevant treatment.</li><li>• This product and related accessories should be handled lightly.</li><li>• Please properly handle the packaging bag of this product to avoid accidents such as asphyxiation of infants, and keep it out of reach of children.</li></ul>

## Components


### ◆ F-362



1. Diamond Dermabrasion
2. Radio Frequency
3. Ultrasonic
4. Scrubber
5. Hot and cold hammer


Main Unit

### Rear Panel

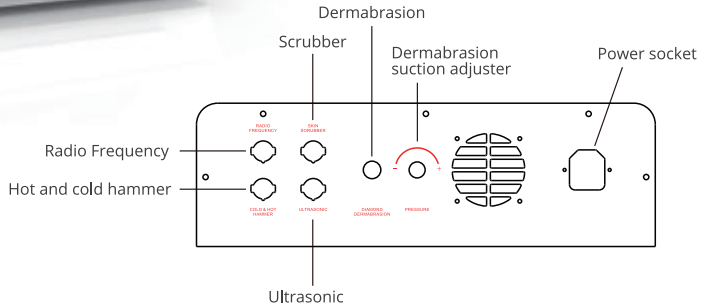

### ◆ Front Panel

1. Hot and cold hammer
2. Ultrasonic
3. Scrubber
4. Radio Frequency
5. Diamond Dermabrasion

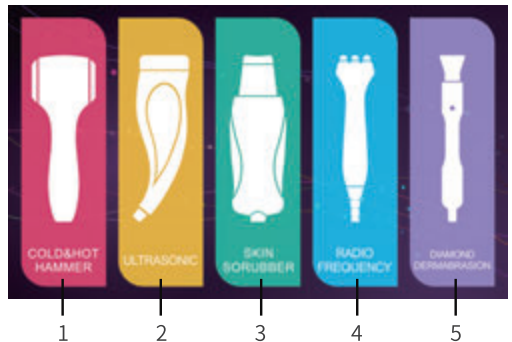

### ◆ Operation Method

#### Operation procedure

- ① Click to choose the required function of main interface, and it will enter the interface of corresponding function.
- ② All working interfaces are with the button to return to the main interface.

## Product Parameters

Product model	F-362
Rated voltage / Frequency	AC 100-240V, 50/60Hz
Power	60W
Net weight	6.1kg
Product size	37×31.5×16.5cm
Package size	56.8×52.5×23.5cm
Package contents	Main Unit × 1
	Accessories Shelf × 1
	Hot and cold hammer × 1
	Ultrasonic probe × 1
	Scrubber × 1
	Radio Frequency × 1
	Diamond Dermabrasion (including 9 pcs) × 1
	Screwdriver × 1
	Screw (M4) × 2
	Manual × 1
	Power cord × 1


## ◆ F-363



1. Spray
2. Vacuum
3. Radio Frequency
4. Ultrasonic
5. Skin Scrubber
6. Hot and cold hammer


Main Unit

### Rear Panel

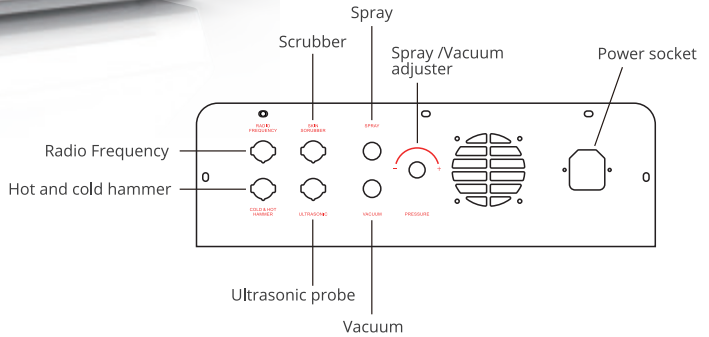

### ◆ Front Panel

1. Hot and cold hammer
2. Ultrasonic
3. Skin Scrubber
4. Radio Frequency
5. Spray /Vacuum

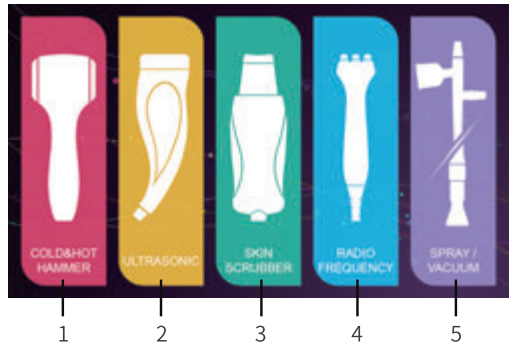

### ◆ Operation Method

#### Operation procedure

- ① Click to choose the required function of main interface, and it will enter the interface of corresponding function.
- ② All working interfaces are with the button to return to the main interface.

## Product Parameters

Product model	F-363
Rated voltage / Frequency	AC 100-240V, 50/60Hz
Power	60W
Net weight	6.2kg
Product size	37×31.5×16.5cm
Package size	56.8×52.5×23.5cm
Package contents	Main Unit × 1
	Accessories Shelf × 1
	Hot and cold hammer × 1
	Ultrasonic probe × 1
	Skin Scrubber × 1
	Radio Frequency × 1
	Vacuum (including 3 Glass suction) × 1
	Sprayer × 1
	Screwdriver × 1
	Screw (M4) × 2
	Manual × 1
Power cord × 1	

## Components


### ◆ F-364



1. Spray
2. Vacuum
3. Radio Frequency
4. Ultrasonic
5. Skin Scrubber
6. Hot and cold hammer


Main Unit

### Rear Panel

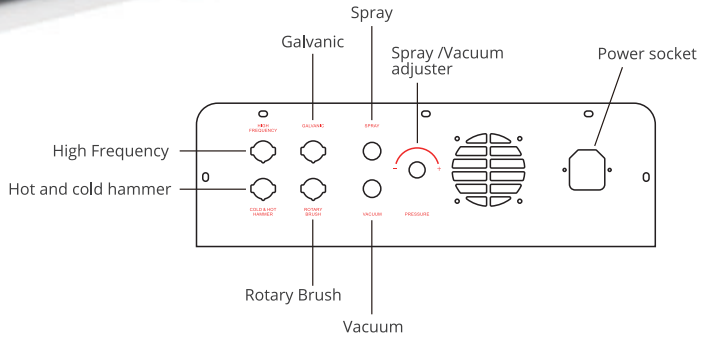

### ◆ Front Panel

1. Hot and cold hammer
2. High Frequency
3. Rotary brush
4. Galvanic
5. Spray / Vacuum

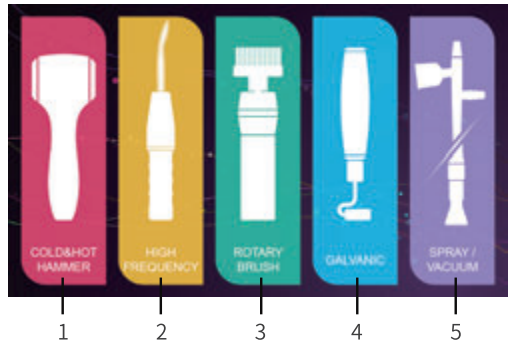

### ◆ Operation Method

#### Operation procedure

- ① Click to choose the required function of main interface, and it will enter the interface of corresponding function.
- ② All working interfaces are with the button to return to the main interface.

## Product Parameters

Product model	F-364
Rated voltage / Frequency	AC 100-120V, 50/60Hz ; AC 220-240V, 50/60Hz
Power	60W
Net weight	6.3kg
Product size	37×31.5×16.5cm
Package size	56.8×52.5×23.5cm
Package contents	Main Unit × 1
	Accessories Shelf × 1
	Hot and cold hammer × 1
	High Frequency probe (including 4 kinds) × 1
	Rotary brush (including 4 kinds) × 1
	Galvanic Probe × 1
	Vacuum (including 3 Glass suction) × 1
	Sprayer × 1
	Screwdriver × 1
	Screw (M4) × 2
	Manual × 1
Power cord × 1	


## ◆ F-365



- 1. Negative plate
- 2. Probe
- 3. CAP handle
- 4. RES handle


Main Unit

## Rear Panel

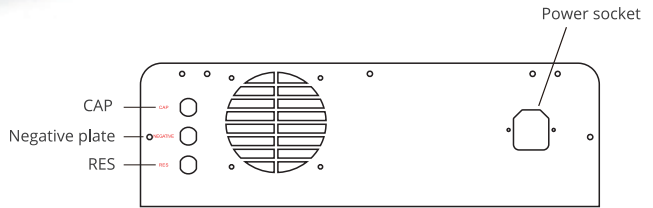

## ◆ Front Panel

- 1. RES
- 2. CAP

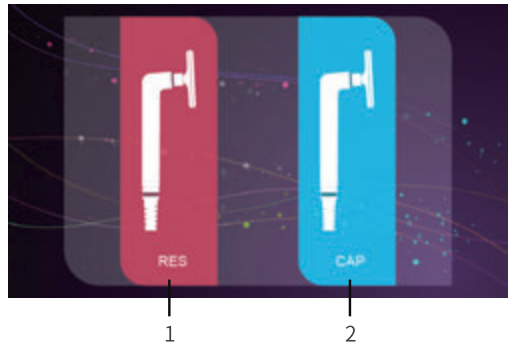

## ◆ Operation Method

### Operation procedure

- ① Click to choose the required function of main interface, and it will enter the interface of corresponding function.
- ② All working interfaces are with the button to return to the main interface.

## Product Parameters

Product model	F-365
Rated voltage / Frequency	AC 100-120V, 50/60Hz ; AC 220-240V, 50/60Hz
Power	400W
Net weight	6 . 8kg
Product size	37×31 . 5×16 . 5cm
Package size	56 . 8×52 . 5×23 . 5cm
Package contents	Main Unit × 1
	Accessories Shelf × 1
	RES handle × 1
	CAP handle × 1
	Negative plate × 1
	Probe × 8
	Screwdriver × 1
	Screw (M4) × 2
	Manual × 1
	Power cord × 1



### Hot and cold hammer

## Operation Cautions

### Warning

- This function is prohibited for the following people or parts :
  - ① Patients with recurrent scar rash and allergic patients;
  - ② Skin burns, scalds, sunburn, inflammation or other wounds;
  - ③ People suffering from immunosuppressive diseases resulting in low immunity, or are using immunosuppressive drugs;
  - ④ There were surgical sites in the last 3 months.
- Beauticians do not leave when the hammer is working, so as not to cause safety accidents.

### Attention


- Hot and cold hammer can only be used indoors, and it is advisable to use indoors at about 25°C. Because its function will be affected by the ambient temperature, such as outdoor use in too high or too low temperature, may not reach the required working temperature, and failed to obtain the expected nursing effect.
- Every time you turn on the beauty instrument and switch the cold and hot function, you need to wait for a moment, until the cold and hot hammer reaches the preset temperature before using. Under normal circumstances, when starting, the time of rising and falling temperature is about 5 minutes; When switching the hot and cold functions, the temperature rises and falls for about 3 minutes.
- When used, can not block the cold and hot hammer around the air slot, so as not to produce condensation water droplets in the cold and hot hammer, and lead to short circuit fault.
- When using, cold and hot hammer should be close to the skin operation, nursing the forehead and frontal bone, the strength should be light.
- When the temperature of hot and cold hammer is high, do not stay still on the skin.
- Do not use hot and cold hammers as percussion tools.
- If there are scratches at both ends of hammer head, please replace with new accessories before use.
- Do not over pull the spring line of hot and cold hammer, pay attention to the range of operable length.

### Recommended using time

Adjust the time according to the specific situation of the nursing site and skin, generally the whole face operation time is 10-15 minutes.



## Hot and cold hammer

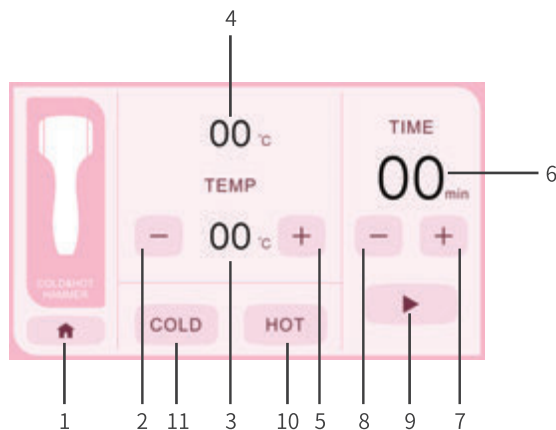

1. Return to main interface
2. Temperature Decreasing Button  
Cooling temperature range: 5~25  
Heating temperature range: 25~45
3. Temperature Display  
When you select the cooling mode, the displayed temperature is the cooling temperature  
When the heating mode is selected, the displayed temperature is the set heating temperature
4. Actual output temperature of Hot & Cold Hammer  
When selecting the cooling mode, the actual output temperature of hammer is displayed.  
When selecting the heating mode, the actual output temperature of the hammer is displayed.
5. Temperature Raising Button  
Cooling temperature range 5C~25° C  
Heating temperature range 25°C ~ 45°C
6. Time Display  
The working time enters the countdown when operating. The machine will stop automatically when the time shows zero.
7. Increasing working time  
Working time range:1~30 minutes.
8. Reducing working time  
Working time range:1~30 minutes.
9. Start/Pause
10. Heating mode
11. Cooling mode

## ◆ Operation Method

- 1) Select cooling mode or heating mode;
- 2) Setting working temperature and working time. The maximum time can be set to 30min.
- 3) Take out the Hot & Cold hammer and turn on the switch of handle.
- 4) Press the start key to start the function
- 5) Check and confirm whether the display temperature is corresponding to setting temperature.  
If yes, touch and feel it with your skin. You can use it directly if you feel it suitable, otherwise please adjust it again.
- 6) The hammer stops automatically when scheduled time ends. Clean it and put it back to the accessories shelf.



### ◆ Ultrasonic probe (Cosmetic use only)

## ◆ Operation Cautions

### Warning

- Do not point the probe directly at the eyeball, touch the eye or massage the upper eyelid.
- It is forbidden for the following groups :
  - ① Pregnant women and those equipped with a cardiac chronometer;
  - ② Tumor patients and serious heart disease;
  - ③ Skin burns, scalds, sunburn, inflammation or other wounds;
  - ④ Allergy patients and people with contact dermatitis.
- It is forbidden to open the probe without cosmetic operation to prevent the risk of high temperature scald and reduce the service life of the probe due to overheating.
- Do not remove the stainless steel probe and handle by yourself. If you turn the stainless steel probe and handle, the probe may be damaged.
- This function shall not be used for other than cosmetic purposes.

### Attention

- Before care, it is necessary to apply enough cream or essence on the care part to ensure that there is a good medium between the probe and the skin, so as not to pull the skin too much.
- When using, the probe should be close to the skin to prevent cavitation and damage to the skin.
- When used, the probe should not be in the output state without operation, in case of damage to the probe or reduce the service life of the probe.
- If the probe is too hot, it will burn the skin, so the use time should not exceed 15 minutes, sensitive skin should reduce the use time and energy intensity. Lengthening the use time will not enhance the nursing effect and may lead to danger. Attention should be paid to the care of a skin stay time should not exceed 5 seconds.
- After nursing, the skin will appear a short fever ruddy phenomenon, due to the vibration of the probe and heat production, increasing blood circulation, which belongs to the normal state.


### Recommended using time

Adjust the time according to the specific situation of the nursing site and skin, generally the whole face operation time is 10-15 minutes.



## Ultrasonic probe (Cosmetic use only)

1. Return to main interface
2. Reducing Energy Level  
Energy level range:1~10
3. Energy Level Display
4. Increasing Energy Level  
Energy level range:1~10
5. Working time display  
The working time enters the countdown when operating. The machine will stop automatically when the time shows zero.
6. Increasing working time  
Working time range: 1~30 minutes.
7. Reducing working time  
Working time range: 1~30 minutes.
8. Start/Pause
9. Continuous Wave Mode  
Output energy is continuous.
10. Intermittent wave mode  
Output energy is intermittent.
11. Mixed Wave Mode  
Switching of continuous wave mode and mixed wave mode.

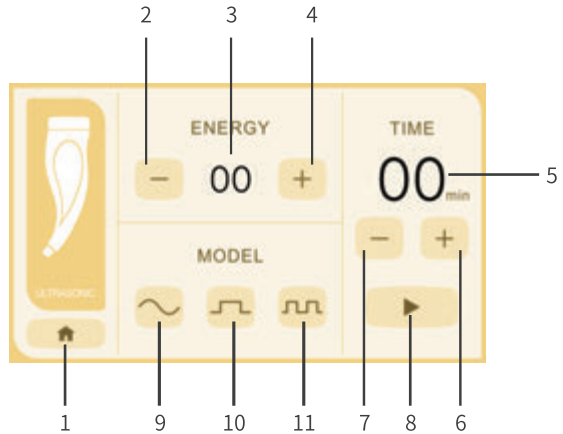

## ◆ Operation Method

- 1) Setting energy output level and working time. The maximum time can be set to 30min.
  - 2) Select energy output mode
  - 3) Take out the ultrasonic handle and turn on the “start” button
  - 4) The ultrasonic stops automatically when scheduled time ends. Clean the metal probe and put it back to the accessories shelf.
- After the nursing operation, the probe shall be disinfected with 75% alcohol and properly dry stored.

Note: Do not wet the metal part of the probe.



Skin Scrubber

### ◆ Operation Cautions

#### Warning

- Do not touch the output interface of Skin Scrubber alone when the host is powered on.
- Do not put the Scrubber directly in contact with the eye.
- It is prohibited for the following people:
  - ① Allergic patients;
  - ② Skin burns, scalds, sunburn, inflammation or other wounds.
  - ③ Such as heart disease, infectious diseases, neurotic patients and patients with metal in the body during treatment.
- Do not put the shovel head of the working scrubber in a part of the skin to avoid damage to the skin.
- When the skin surface of the skin cream is not enough to moisturize the skin, please timely supplement.
- When using the scrubber, pay attention to the strength and the angle of the scrubber and the skin.
- When working, pay attention to the connection between the accessories and the host. Do not pull excessively, so as not to cause the connection line to fall off.
- When working, pay attention not to be splashed with water or other corrosive liquids.
- Do not use the stainless steel surface has been deformed defect spatula, so as not to lose skin.
- If there is any abnormal situation, please contact the dealer and ask professional maintenance personnel to check, do not open the chassis for maintenance without authorization.

#### Attention

- Keep the scrubber clean at all times and disinfect it with 75% alcohol after each use.
- The stay time of one area of skin should not exceed 5 seconds.

#### Recommended using time


Generally the whole face operation time is 15 minutes.

- After the nursing operation, sterilize the spatula and antistatic wrist cover with 75% alcohol, and store in a dry place.

Do not wet the metal part of the ESD wrist strap.



Skin Scrubber

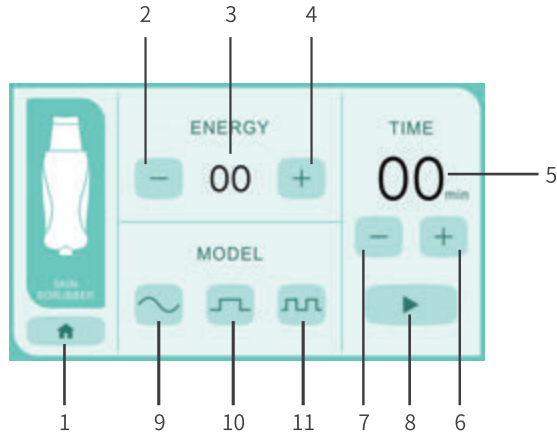

1. Return to main interface
2. Reducing Energy Level  
Energy level range: 1~10
3. Energy Level Display
4. Increasing Energy Level  
Energy level range: 1~10
5. Working time display

The working time enters the countdown when operating.  
The machine will stop automatically when the time shows zero.

6. Increasing working time  
Working time range: 1~30 minutes.
7. Reducing working time  
Working time range: 1~30 minutes.
8. Start/Pause
9. Continuous Wave Mode  
Output energy is continuous.
10. Intermittent wave mode  
Output energy is intermittent.
11. Mixed Wave Mode  
Switching of continuous wave mode and mixed wave mode.

## ◆ Operation Method

- 1) Set energy gear and working time, the longest time can be set for 30min;
- 2) Select the output mode;
- 3) Take out the Skin Scrubber and press the start button to use;
- 4) After the working time ends, clean and wipe the Skin Scrubber and put it back to the accessory shelf.

Note: add a small amount of water to the metal part of the Skin Scrubber.  
When the Skin Scrubber works, the water will evaporate immediately and form mist.



### Radio Frequency

## Operation Cautions

### Warning


- Please use the product at the rated voltage conforming to the product and ensure that the plug is fully inserted, otherwise electric shock or fire may occur.
- Do not use it in the bathroom or in a damp room as it may cause electric shock or fire.
- Do not use in paint, coating, oxygen supply and other flammable and explosive places.
- If you do not use this function, press the pause button.
- This function should not be used for patients with heart disease, diabetes, hyperthyroidism, malignant tumor, severe hypertension, implanted metal or pacemaker, epilepsy, skin and skin cancer. When used by persons with physical, sensory or mental disabilities or those lacking experience and knowledge (including children), a guardian must supervise and guide the use of the product and be responsible for the safety of the user. Please prevent children from handling the product and treating it as a toy.
- Pregnant women, lactation women, menstrual women should not use.
- Do not use on the injured parts and joints, so as not to affect wound healing.
- This function shall not be used for other than cosmetic purposes.

### Attention

- It is recommended that the stay time of one skin should not exceed 5 seconds to avoid scalding.
- The probe should avoid the eyeball and thyroid gland when nursing.
- After the nursing operation, the probe should be disinfected with 75% alcohol, and properly dry storage.
- This function is not recommended for areas where artificial fillers, such as silica gel, have been injected.
- It is recommended to remove all metal accessories before using this function.



## Radio Frequency

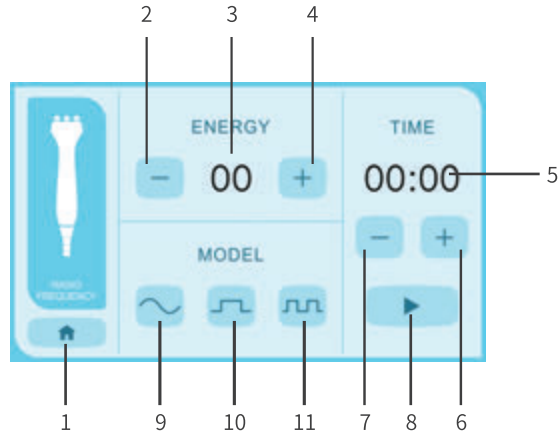

1. Return to main interface

2. Reducing Energy Level

Energy level range: 1~10

3. Energy Level Display

4. Increasing Energy Level

Energy level range: 1~10

5. Working Time Display

The working time enters the countdown when operating.

The machine will stop automatically when the time shows zero.

6. Increasing working time

Working time range: 1~30 minutes.

7. Reducing working time

Working time range: 1~30 minutes.

8. Start/Pause

9. Continuous Wave Mode

Output energy is continuous.

10. Intermittent wave mode

Output energy is intermittent.

11. Mixed Wave Mode

Switching of continuous energy mode and mixed wave mode.

## ◆ Operation Method

1) Set energy gear and working time, the longest time can be set for 30min;

2) Select the output mode;

3) Take out the RF accessory and press the Start button to use it;

4) After the end of the working time, clean the Radio Frequency metal head and put it back to the accessory shelf.



### ◆ Dermabrasion

## ◆ Operation Cautions

### Warning


- This function is not applicable to patients with hypertension, heart disease and diabetes.
- Pregnant women, lactation women, menstrual women should not be used.
- It is prohibited for use by patients with infectious diseases and cancer.
- Botox is banned for people who have received injections within a year.
- People with scar tissue and collagen deficiency should not use this function, otherwise it will cause permanent skin damage.
- This function shall not be used for other than cosmetic purposes.
- Prevent children from being exposed to the product and treating it as a toy, thereby causing danger.

### Attention

- Analyze the user's skin condition before care, and then select the right grinding head and proper suction.
- It is recommended to clean the skin and dry the face of excess moisture before care.
- After care, clean the face with water.
- After the nursing operation, the grinding head should be disinfected with 75% alcohol and stored properly and dry.



## Dermabrasion

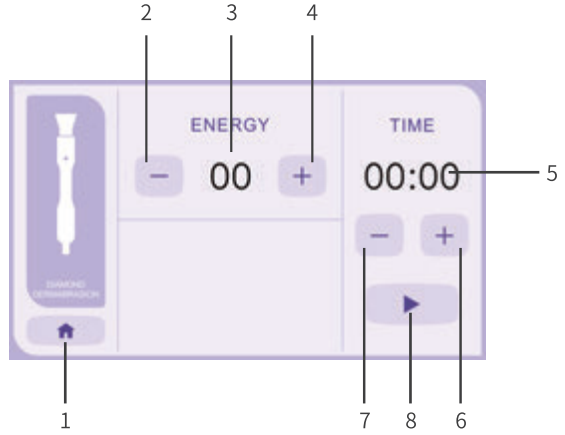

1. Return to main interface
2. Reducing Energy Level  
Energy level range:1~10
3. Energy Level Display
4. Increasing Energy Level  
Energy level range:1~10
5. Working Time Display

The working time enters the countdown when operating.

The machine will stop automatically when the time shows zero.

6. Increasing working time  
Working time range:1~30 minutes.
7. Reducing working time  
Working time range:1~30 minutes.
8. Start/Pause

## ◆ Operation Method

- 1) Set energy gear and working time, the longest time can be set for 30min;
- 2) Take out the accessories and install the grinding head needed;
- 3) Press the Start button to use;
- 4) After the end of the working time, clean the accessories and put them back to the accessory shelf.

## Function Introduction



Sprayer



Suction

## ◆ Operation Cautions

### Warning

- Do not operate suction function on eyes and surrounding skin.
- The following people are prohibited to use suction and spray functions:
  - ① Allergic patients;
  - ② Skin burns, scalds, sunburn, inflammation or other wounds.

### Attention

- If there is no suction when the machine is started, it is a normal phenomenon. Wait for a moment and it can be restored to normal.
- Before each use, check whether the glass straw is cracked or broken. In such case, it should be prohibited to use and replace the glass straw immediately
- Glass straw can not be used in the same part for too long. Move suction pen fast, so as not to cause skin healing.
- Do not misconnect the nozzle into the suction jack, in case the skin care products are sucked into the air pump and the main machine is damaged.
- Do not pull the airway vigorously to avoid damage.
- After use, the power must be cut off.




Sprayer



Suction

1. Return to main interface
2. Reducing Energy Level  
Energy level range: 1~10
3. Energy Level Display
4. Increasing Energy Level  
Energy level range: 1~10
5. Working Time Display  
The working time enters the countdown when operating.  
The machine will stop automatically when the time shows zero.
6. Increasing working time  
Working time range:1~30 minutes.
7. Reducing working time  
Working time range: 1~30 minutes.
8. Start/Pause
9. Vacuum mode
10. Spray mode

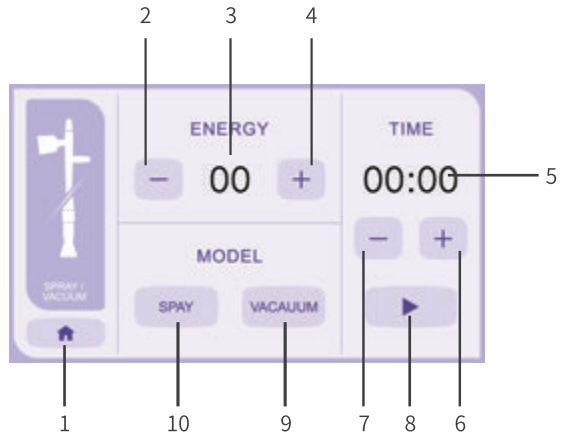

## ◆ Operation Method

- 1) Choose the water refill instrument or suction function. If you use the water refill instrument, you must pay attention to whether the water volume of the water bottle is sufficient;
- 2) Set energy gear and working time, the longest time can be set for 30min;
- 3) Take out the accessories and install the suction head needed;
- 4) Press the Start button to use;
- 5) After the end of the working time, clean the accessories and put them back to the accessory shelf.

### Recommended using time

Under normal circumstances, the continuous operation time of suction and spraying should not exceed 10 minutes.



### High frequency electrotherapy


## Operation Cautions

### Warning


- It is forbidden to touch the output interface of the main machine of the high-frequency electrotherapy beauty instrument or the output interface of the handle of the high-frequency electrotherapy when the main machine is powered on.
- Do not point the tube directly or in contact with the eyes area.
- This product is prohibited for the following people and parts:
  - ① Pregnant women and those equipped with adjusters (e.g. cardiac adjusters).
  - ② People with vascular diseases and high skin sensitivity.
  - ③ Having metal implants or metal dentures embedded in the body.
  - ④ People with skin wounds and other conditions.
  - ⑤ Body parts with thick hair.
- Do not use skin lotions containing alcohol, as alcohol is highly flammable. If alcohol-based toner is required, only apply it after using this product.
- Do not grasp the junction between the high-frequency handle and the electrode tube by hand during operation to avoid being shocked.
- The skin retention time of each part should not exceed 5 seconds.
- This function shall not be used for other than cosmetic purposes.

### Attention

- Before use, check whether the electrode tube or the metal head on the tube is loose and whether the electrode tube cracks or is damaged. If so, do not use it and replace the electrode tube immediately.
- The current intensity must be gradually reduced to zero and the power switch must be turned off before the electrode tube can be inserted or pulled out.
- When inserting or pulling out the electrode tube, first wear rubber gloves and hold the electrode tube with thumb and forefinger: insert the tube to the bottom of the hole until it is firm, and do not push the tube with the palm to prevent the glass from breaking and causing injuries; It is forbidden to pull out vigorously, so as to prevent the metal head on the electrode tube from falling off or breaking, which may cause injury.

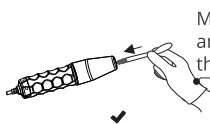

MUST wear the rubber gloves, and hold with thumb and forefinger, then insert the electrode pipe till firmly.

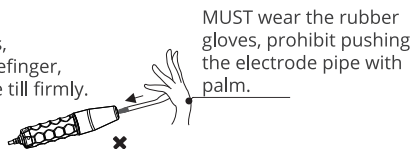


MUST wear the rubber gloves, prohibit pushing the electrode pipe with palm.

- When in use, the high voltage current output from the electrotherapy tube will cause interference.
- It is forbidden to approach the host machine and other electrical appliances of the high-frequency electrotherapy beauty instrument. If it interferes with the host, it may cause the host to not work or crack.
- When the electrode tube is less than 2.5cm from the skin but not in contact with the skin, the current intensity should be reduced immediately if sparks emits to irritate the skin. If the customer still feels the spark after the above operation, a small piece of dry cotton towel (nylon or other synthetic fiber towels are not allowed) put on the customer's face.
- When using, always keep the electrode tube on the customer's skin. High-frequency indirect care: when turning on and off the instrument, one hand must always be in contact with the customer's skin, never leave the customer's skin. Both sides feel the current and it causes discomfort.
- After use, the power must be turned off first, the electrode tube should be pulled out from the high-frequency handle and after disinfection and drying put it into the accessory box.



## High frequency electrotherapy

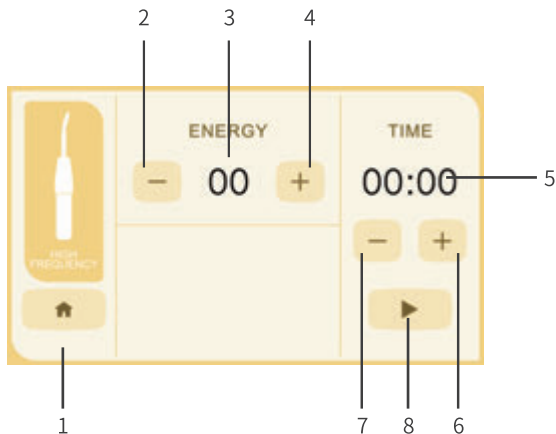

1. Return to main interface

2. Reducing Energy Level  
Energy level range:1~10

3. Energy Level Display

4. Increasing Energy Level  
Energy level range:1~10

5. Working Time Display

The working time enters the countdown when operating.  
The machine will stop automatically when the time shows zero.

6. Increasing working time  
Working time range:1~30 minutes.

7. Reducing working time  
Working time range:1~30 minutes.

8. Start/Pause

## ◆ Operation Method

- 1) Set the output gear and working time, the longest setting can be 30min;
- 2) Take out the handle and install the electrotherapy tube needed;
- 3) Press the Start button to enable the function;
- 4) After the scheduled end, the accessories stop working, take out the electrotherapy tube, clean and wipe the high frequency electrotherapy accessories probe, and then put it back to the accessories shelf.

Note: Do not wet the metal parts of the electrotherapy tube.

### Recommended using time

- ① Oily skin: recommended to use 10-15 minutes;
- ② Dry and dehydrated skin: 3-5 minutes is recommended.



Rotary brush

### ◆ Operation Cautions

#### Warning


- Do not operate under the eyes.
- The following groups are forbidden to use the brush function:
  - ① Allergic patients;
  - ② Skin burns, scalds, sunburn, inflammation or other wounds.

#### Attention

- Before use, check whether the brush head is firmly inserted into the brush handle.
- Do not use the brush while holding the main brush head.
- Do not plug or replace the brush head during use.
- Do not use your hands or other objects to force the brush to turn.
- Do not use the brush function for a long time to avoid overheating of the brush handle.
- Turn the brush head down before touching the customer's skin. When touching customer's skin, the force should be light, so as not to cause customer discomfort.



Rotary brush

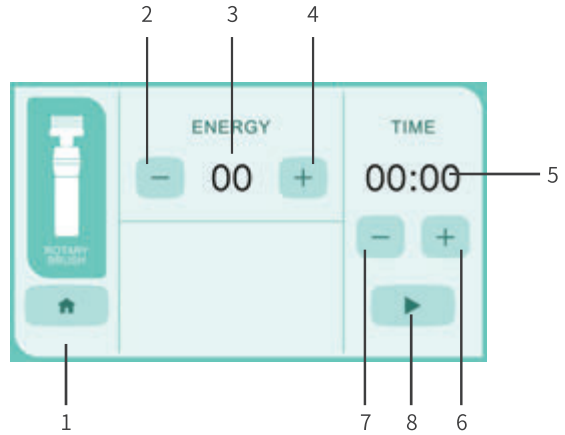

1. Return to main interface
2. Reducing Energy Level  
Energy level range: 1~10
3. Energy Level Display
4. Increasing Energy Level  
Energy level range: 1~10
5. Working Time Display

The working time enters the countdown when operating.  
The machine will stop automatically when the time shows zero.

6. Increasing working time  
Working time range: 1~30 minutes.
7. Reducing working time  
Working time range: 1~30 minutes.
8. Start/Pause

## ◆ Operation Method

- 1) Set energy gear and working time, the longest setting can be 30min;
- 2) Take out the accessories and install the brush head needed;
- 3) Press the Start button to use;
- 4) After the end of the working time, clean the accessories and put them back to the accessory shelf.

### Recommended using time

Adjust the duration according to the specific conditions of the skin. Generally, the duration should not exceed 10 minutes.



Galvanic roller bar



Galvanic hand-held stick



## ◆ Operation Cautions

### Warning

- Do not operate the roller directly to the eyeball or directly contact the eye.
- It is forbidden to use this product for the following people and parts:
  - ① Pregnant women and those with metal substances in their bodies;
  - ② Allergic patients and tumor patients;
  - ③ Skin burns, scalds, sunburn, inflammation or other wounds;
  - ④ People with dilated capillaries.
- It is forbidden to import with strong alkaline solution to prevent negative chemical damage.
- This function shall not be used for other than cosmetic purposes.

### Attention

- The acidity and alkalinity of skin care products should be used with the positive and negative poles of the roller. When the roller is positive, the lead-in liquid is acidic (+); When the roller is negative, the import liquid is alkaline (-), such as the choice of skin care products are not directly marked acid alkaline, can buy PH test paper to determine.
- When using, do not contact the roller directly with the hand-held stick. If contact is careless, the main machine will enable automatic protection and disconnect the output.
- Customers with thick cuticle should first exfoliate, and then apply the imported material evenly to the operation site for import operation.
- Different substances (two or more chemicals) cannot be introduced together or successively into the same site, regardless of whether they have the same charge.
- Under normal circumstances, the effect of ion import is inversely proportional to the molecular weight of the imported substance, that is, the larger the molecular weight, the worse the import effect, and the smaller the molecular weight, the better the import effect.

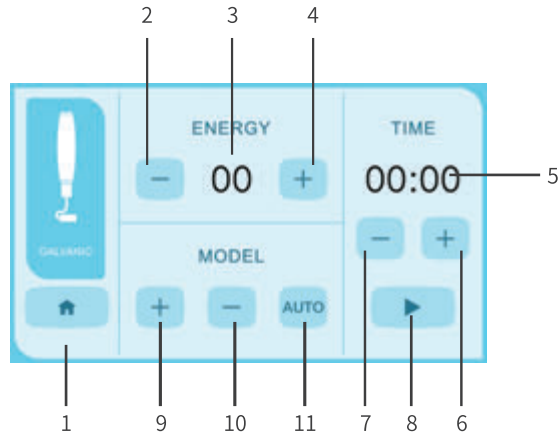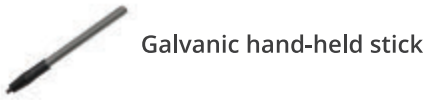

1. Return to main interface
2. Reducing Energy Level  
Energy level range: 1~10
3. Energy Level Display
4. Increasing Energy Level  
Energy level range: 1~10
5. Working Time Display  
The working time enters the countdown when operating.  
The machine will stop automatically when the time shows zero.
6. Increasing working time  
Working time range:1~30 minutes.
7. Reducing working time  
Working time range:1~30 minutes.
8. Start/Pause
9. Positive micro current output mode  
output positive micro current signal.
10. Micro current negative output mode  
output micro current negative signal.
11. The mixed output mode periodically switches between positive and negative micro current output.

## ◆ Operation Method

- 1) Set energy gear and working time, the longest setting can be 30min;
- 2) Select the output mode;
- 3) Take out the Galvanic accessory and let the guest hold the stick with his hand;
- 4) Press the Start button to use;
- 5) After the end of the working time, clean the accessories and put them back to the accessory rack.

### Recommended using time

- ① The duration of penetration care: anionic operation for 2 minutes, then cationic operation for 6 minutes;
- ② Descaling care duration: anionic operation for 6 minutes, and cationic operation for 2 minutes.

## Function Introduction



RES mode



CAP mode



Negative Plate

## Operation Cautions

### Warning

- This function is not suitable for patients with hypertension, heart disease and diabetes
- Not suitable for lactating women and menstruating women.
- It should not be used by patients with infectious diseases or tumors.
- The function should not be used for any purpose other than cosmetic.
- Prevent children from being exposed to the product and treating it as a toy, thereby causing danger.
- During use, the energy probe should not stay in the same position to avoid scalding.

### Attention


- Before care, pay attention to whether the connection wire, power cord socket and probe are fully inserted and installed.
- Select the appropriate gear and probe before care.
- Do not stay for a long time to take care of one area of skin to avoid scalding. .
- After the nursing operation, the probe should be disinfected with 75% alcohol and properly stored dry.
- Relevant gel or essence must be applied to the body parts to moisturize the skin.
- Make all electrodes completely contact with the guest's skin with moderate force to avoid skin tingling or red spots caused by current aggregation.
- Equipment with human impedance identification function, only after good contact, can start output.

## Installation Precautions

- The F-365 Multi-function beauty instrument includes one negative plate, two handles and two probes.
- RES resistance mode handles are marked with a red circle and must be fitted with stainless steel probes only.
- CAP Capacitive mode handle with blue circle logo must be mounted with black probe only.
- The pins of the two probes have different diameters, so it is necessary to install the probe corresponding to the handle correctly. If the probe is installed incorrectly, it will lead to the probe cannot be installed or the probe is loose and easy to fall off, so it cannot be matched.



## RES mode

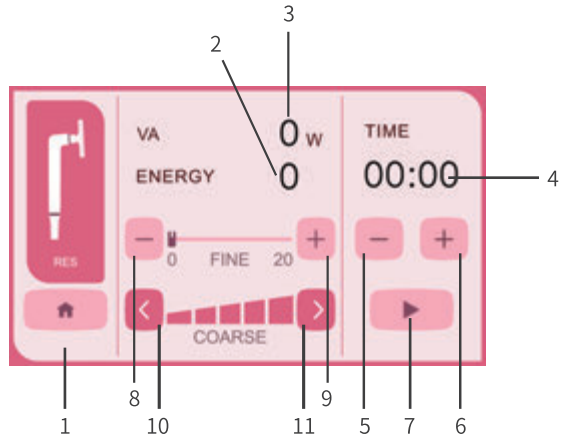


1. Return to the main interface
2. Real-time power display  
handle and negative plate form a loop to display the actual output power.
3. Output range display  
Displays the set range from 0 ~ 100.
4. Working time display  
When starting work, the working time enters the countdown.  
When the countdown is zero, the device automatically stops output.
5. Reduce the working time  
Working time range: 1 ~60 minutes.
6. Increase the working time  
Working time range: 1 ~60 minutes
7. Start/pause
8. The output gear is reduced by fine tuning  
Adjustable 20 gears
9. The output gear is slightly increased  
Adjustable 20 gears.
10. Coarse shift reduced.  
Adjustable 5 gears, each gear can be fine-tuned 20 gears.
11. Coarse shift increased  
Adjustable 5 gears, each gear can be fine-tuned 20 gears.

## ◆ Operation Method

- 1) Set timing time and output gear;
- 2) Remove the handle and install the stainless steel probe;
- 3) After the guests touch the negative plate, press the start button to open the function;
- 4) After the end of the timing, the probes will stop working, remove the probe, then wipe clean, and put the handle and probe back to the accessory shelf.



## CAP mode

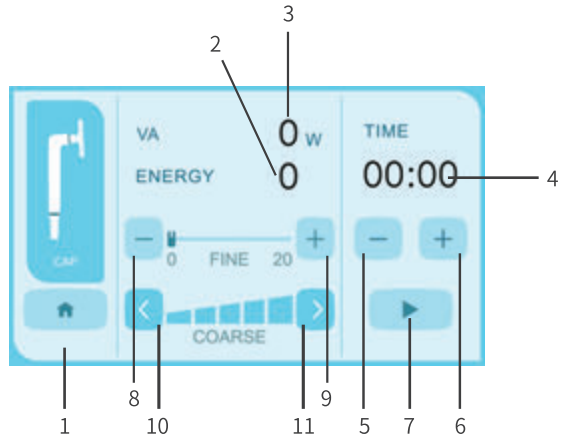

1. Return to the main interface
2. Real-time power display  
Handle and negative plate form a loop to display the actual output power.
3. Output range display Displays the set range from 0 to 100.
4. Working time display  
When starting work, the working time enters the countdown.  
When the countdown is zero, the device automatically stops output.
5. Reduce the working time  
Working time range: 1 ~60 minutes.
6. Increase the working time  
Working time range: 1 ~60 minutes
7. Start/pause
8. The output gear is reduced by fine tuning,  
Adjustable 20 gears
9. The output gear is slightly increased,  
Adjustable 20 gears.
10. Coarse shift reduced.  
Adjustable 5 gears, each gear can be fine-tuned 20 gears.
11. Coarse shift increased  
Adjustable 5 gears, each gear can be fine-tuned 20 gears.

## ◆ Operation Method

- 1) Set timing time and output gear;
- 2) Remove the handle and install the black probe;
- 3) After the guests touch the negative plate, press the start button to open the function;
- 4) After the end of the timing, the attachment will stop working, remove the probe, then wipe clean, and put the handle and probe back to the accessory shelf

## Troubleshooting

The Fault Phenomenon	Reason	Solution
The main machine does not work or crashes.	1. No power.	1. Check whether the fuse in the fuse bin in the power base is burned out. If burned out, replace the corresponding specifications of the new fuse. 2. Press the start button, the button will light up after power.
	2. The power cord is not properly inserted or connected.	2. Plug or connect the power cord properly.
Display screen doesn't work.	1. No power.	1. Turn on the power.
Accessory probe doesn't work.	1. Poor interface contact.	1. Unplug the output cable, and then reinsert the socket.
Weak output for Cold and hot hammer, ultrasonic, scrubber, RF, Galvanic and other accessories.	1. The tap position is not set well; 2. The high temperature of the accessories leads to low output power.	1. Adjust the high-grade position; 2. Appropriate use, do not open for a long time. Turn off the power switch when accessories are not in use.
Dermabrasion has weak suction.	1. The tap position is not set well; 2. The inside of the Dermabrasion is blocked; 3. Aging of air tube; 4. The interface between the air tube and the suction pen is loose.	1. Adjust the gear position and adjust the air regulating valve; 2. Replace the Dermabrasion accessories; 3. Replace the new air tube and insert the interface.
Sprayer with weak spraying effect.	1. The tap position is not set well; 2. The water refill instrument is blocked inside; 3. Aging of air tube; 4. The interface between the air tube and the water refill instrument is loose; 5. The water in the bottle is insufficient.	1. Adjust the high-grade position; 2. Replace the accessories of the water refill instrument; 3. Replace the new air tube and insert the interface; 4. Pay attention to the content of water in the water bottle and add water in time.
The suction of the pen is weak.	1. The tap position is not set well; 2. The pen suction tube is blocked inside; 3. Aging of air tube; 4. The interface between the air tube and the suction pen is loose.	1. Adjust the high-grade position; 2. Replace the suction pen attachment; 3. Replace the new air tube and insert the interface.
The output of high frequency electrotherapy is weak.	1. The tap position is not set well; 2. Aging after long-term use of electric therapy rod.	1. Adjust the high-grade position; 2. Replace the electrotherapy accessories.
The host jitter.	1. Accessories are blocked.	1. Check whether the air tube of relevant accessories is blocked.

If problem is still not solved after above operations, please contact maintenance center or supplier to solve it.



Do not dispose of old or defective appliances in domestic garbage; this should only be done through public collection points.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.