ES - EN

WK_S013
Manual de instrucciones
Instruction manual
WE-2018-WK_S013

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
PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	Extra-large stones (sacrum and solar plexus)	-	2
2	Large stones (back and thighs)	-	14
3	Medium stones (hands, arms, calves, feet and nape)	-	14
4	Small stones (forehead, top and back of the head)	-	7
5	Toe stones	-	8
6	Deep tissue stones	-	4
7	Contour ones (for the eyes)	-	2
8	Marble stones for cold therapy	-	6
9	Chakra stone	-	7

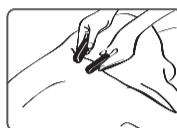INSTRUCTIONS OF USE

1. Pour water into the removable water reservoir until you cover the stones.

2. Firmly close the heater's lid.

3. Set the temperature through the electronic screen and it will soon achieve the desired temperature.

4. Once the stones are hot, pick them up with the slotted spoon provided.

5. To heat stones, you should take into account that warm stones would range between 43°C and 48°C, and hot stones between 45°C and 58°C, depending on the professional's preference for the treatment.

6. Only use professional stone heaters to heat the stones. Other equipment which is not specially conceived for this use, such as electric cookers or ovens, can overheat the stones and may injure the patient's skin or damage the stones.

CAUTIONS

1. Keep the stones out of children reach.

2. Avoid using the stones with children or people that may have a lack of sensitivity to high temperatures, such as people suffering diabetes.

3. Do not use them with pregnant women or any patient suffering from heart condition, thrombosis, tuberculosis, any tumour or any patient having varicose veins, bruises, rashes, swelling areas or any lesion.

4. If the patient is following any other treatment against pain, please ask the doctor before applying these hot stones on the area.

5. In order to prevent the patient from burns, test the temperature of the stones on your hands before applying them on anyone else. Check together with the patients if the temperature is proper for them.

MAINTENANCE

It is very important to always keep the stones clean so that every massage session is as safe and hygienic as the first day you used them. In order to clean them, you can put the stones inside the stone heater in water around 65°C for 15 minutes and dry them with a clean towel before storing them again in the bamboo box.

If you do not want to use the stone heater to clean them, you can rinse them with cold running water, and then use some antibacterial soap and warm water to get rid of any oil, cream or remaining lotion and rinse them again. Dry them with a lint-free cloth and place them back into their case.

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ENGLISH

WARNING

Please read this document carefully before using the device. Keep it in an accessible place so that you can read it at any time. You will be working with high-temperature stones in contact with the body, so it is very important to follow the next guidelines in order to avoid any injury. Take into account that these massage stones ref. BREATHE are conceived for being heated in the Weelko stone heaters (ref. CHILL and ref. HARMONY).

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	49x22x8
	Gross weight (kg)	8
	Product size (cm)	46,5x20x7,7
	Net weight (kg)	7,7

© 2018 Weelko

Visit our website at www.weelko.com for more information

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